

Deltec Cozmo[®] Insulin Pump Basics

Model 1700



Left programming key

To select any action listed in the left-hand corner of the screen, press the left button on the key pad

Right programming key

To select any action listed in the right-hand corner of the screen, press the right button on the key pad

Up/down arrow keys

Push to change a highlighted or 'boxed' word/number



Screen saver/sleep mode

- Time and current basal rate is displayed
- Turn back light on by pushing the down arrow button




Home screens

All home screens show:

- Amount of insulin remaining in the cartridge (top left)
- Amount of battery life remaining (top right)
- Stop (left button)
- Menu (right button)

Menu Map for the Deltec Cozmo® Insulin Pump Model 1700

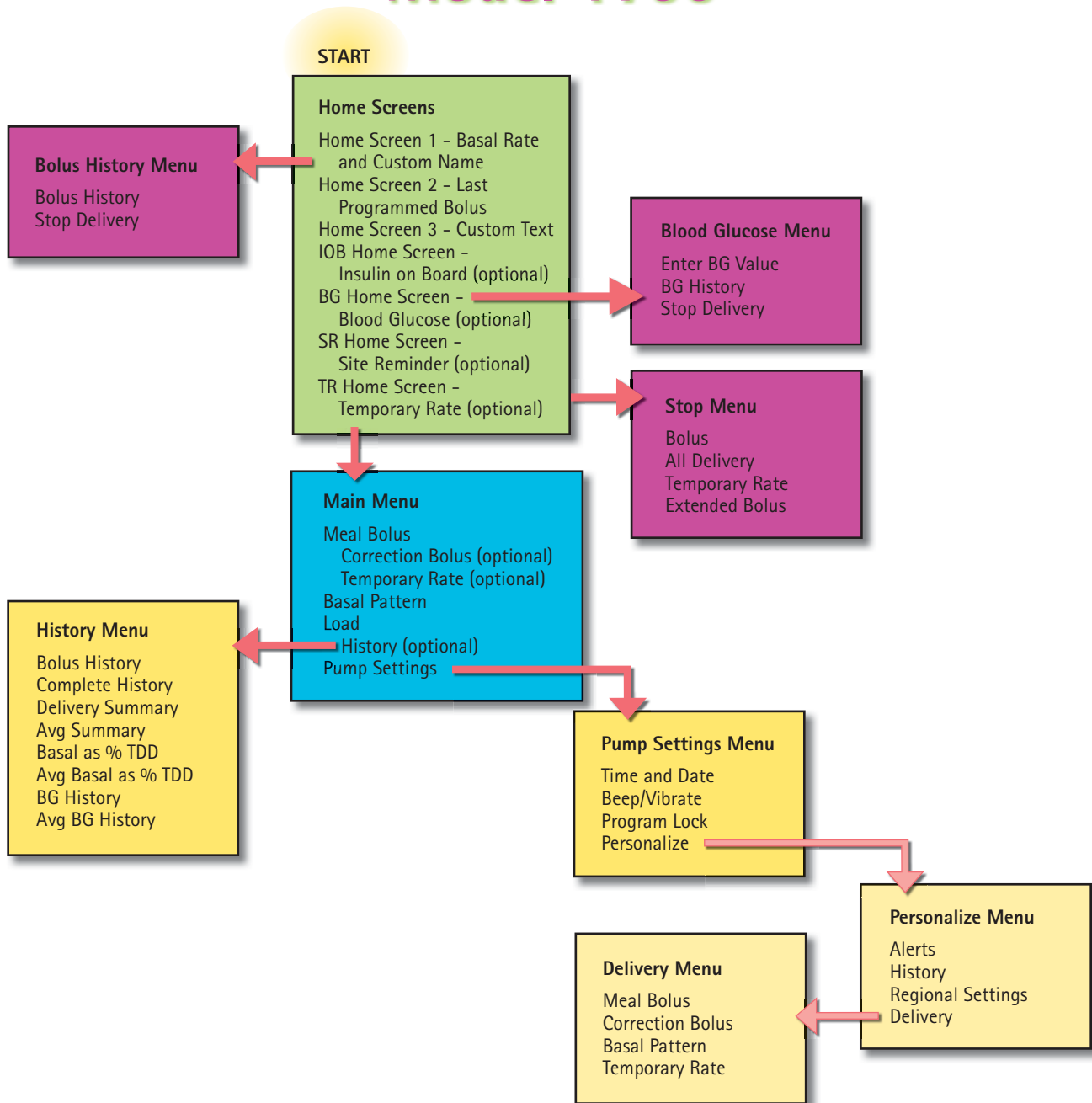

smiths medical

Smiths Medical MD, Inc.

St. Paul, MN 55112, USA

Phone: 1-800-426-2448

www.smiths-medical.com

Smiths Medical, part of the global
technology business Smiths Group

www.cozmore.com

CozMore, Deltec Cozmo, CoZmonitor, Meal Maker, Hypo Manager and the Smiths Medical design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2008 Smiths Medical family of companies. All rights reserved. 8/08 191943

CozMore
Insulin Technology System