

Deltec Cozmo[®] Insulin Pump Basics

Model 1800



Left programming key

To select any action listed in the left-hand corner of the screen, press the left button on the key pad

Right programming key

To select any action listed in the right-hand corner of the screen, press the right button on the key pad

Up/down arrow keys

Push to change a highlighted or 'boxed' word/number



Screen saver/sleep mode

- Time and current basal rate is displayed
- Turn back light on by pushing the down arrow button




Home screens

All home screens show:

- Amount of insulin remaining in the cartridge (top left)
- Amount of battery life remaining (top right)
- Stop (left button)
- Menu (right button)

Menu Map for the Deltec Cozmo® Insulin Pump Model 1800

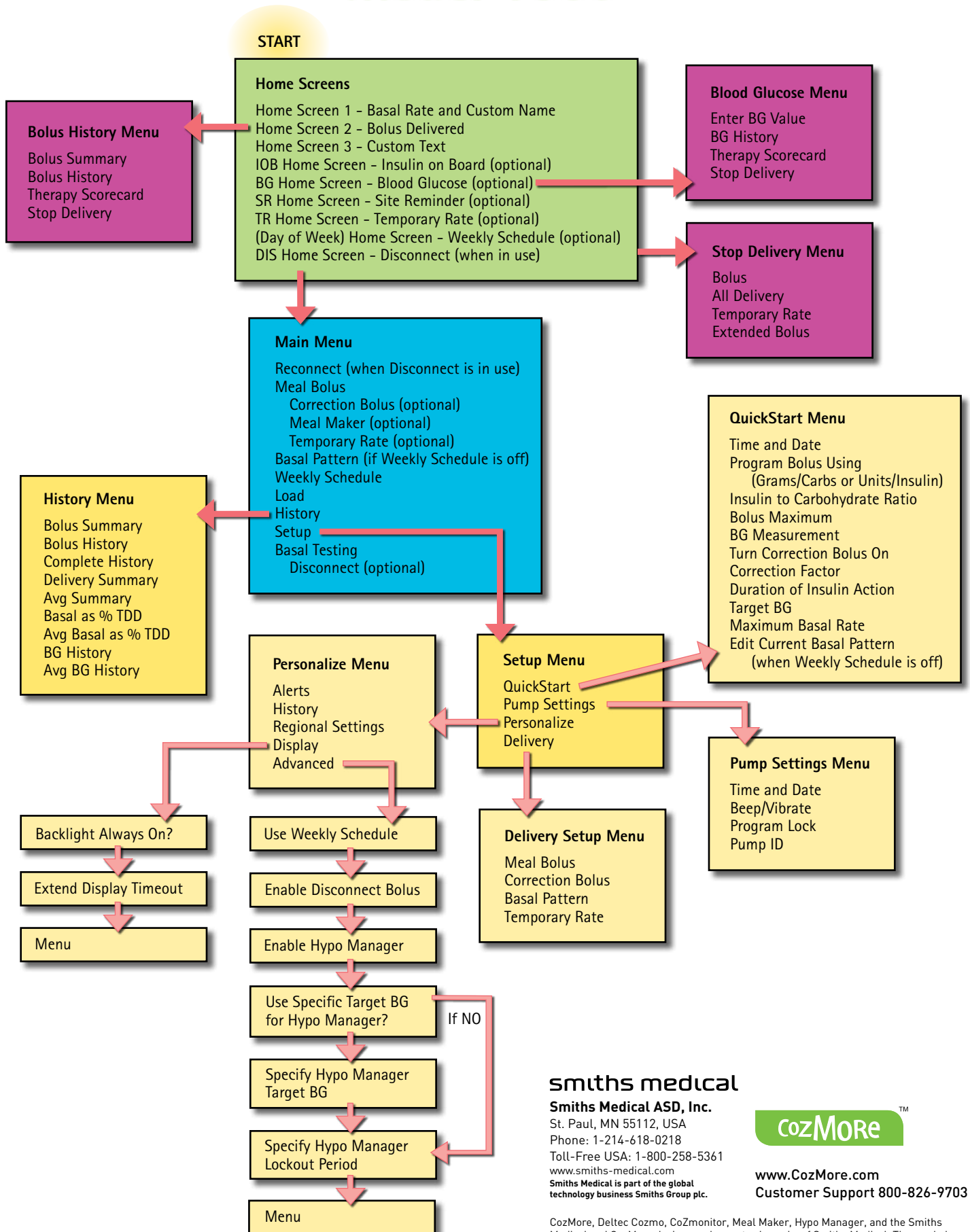

smiths medical

Smiths Medical ASD, Inc.

St. Paul, MN 55112, USA
 Phone: 1-214-618-0218
 Toll-Free USA: 1-800-258-5361
 www.smiths-medical.com
 Smiths Medical is part of the global
 technology business Smiths Group plc.



www.CoZmore.com
 Customer Support 800-826-9703

CozMore, Deltec Cozmo, CoZmonitor, Meal Maker, Hypo Manager, and the Smiths Medical and CozMore design marks are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2011 Smiths Medical. All rights reserved. 2/11 DI191942