


Medfusion® 3000 Series Lockbox

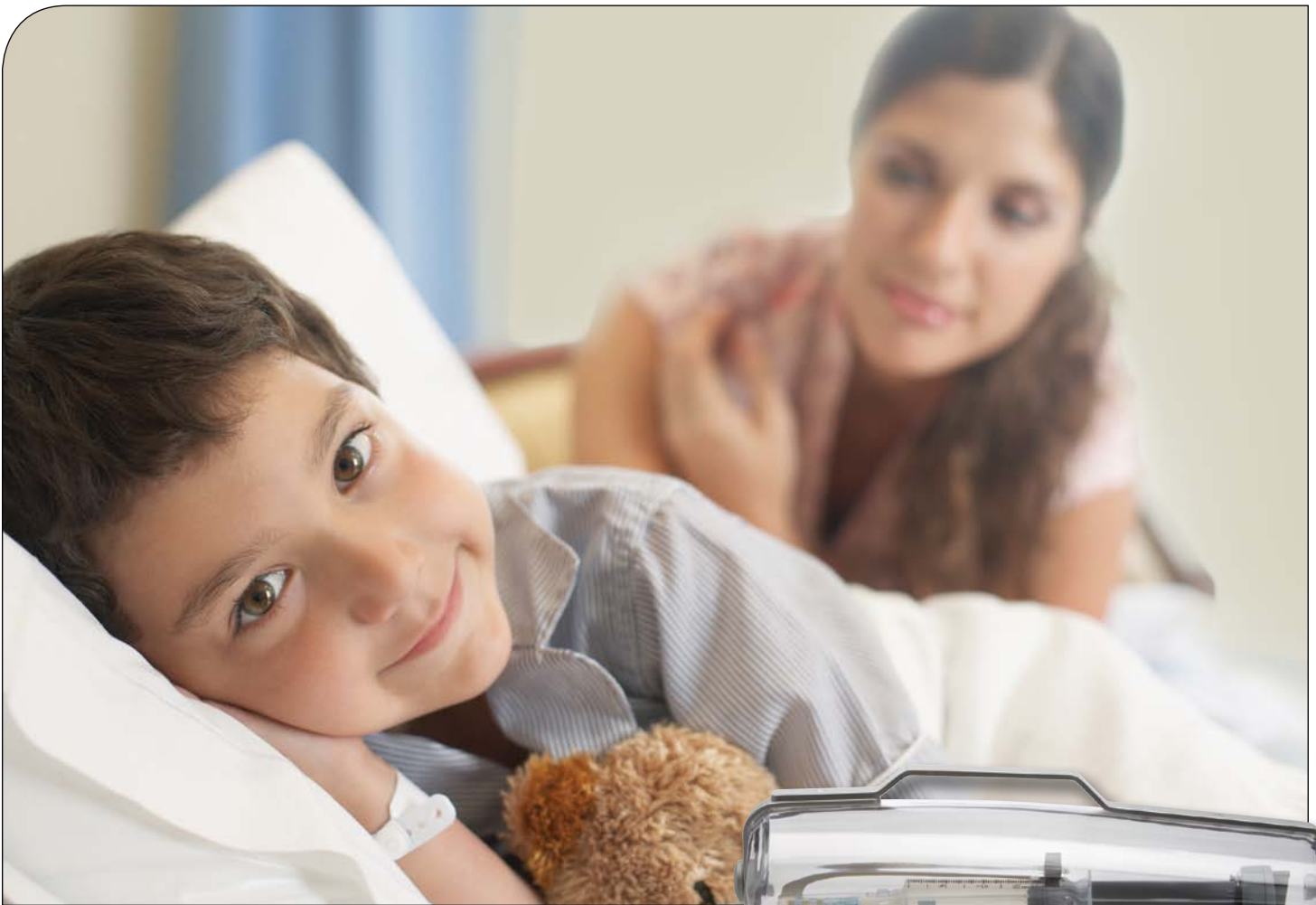

**Additional security for Medfusion®
syringe pumps designed to promote
patient safety and limit syringe access**



Medfusion® 3000 Series Lockbox

The new Medfusion® 3000 Series lockbox provides additional security to help prevent access to the medication and mechanisms on Medfusion® 3000 series syringe pumps. The thermoformed, clear plastic provides exceptional visibility of the pump's syringe and syringe label to ensure accurate medication delivery for your patients.

The Medfusion® Lockbox:

- Locks with a key to secure the syringe, tubing connection and syringe-mounting and driving mechanisms
- Allows access to the pump's display and keypad, which can be locked via the pump's lock options
- Can be mounted on a shelf, IV pole, or horizontal rectangular rail



Easy Installation – No Tools Required

A Medfusion® 3000 Series syringe pump equipped with the mounting bracket for the rotating pole clamp can be easily installed into the lockbox without the use of tools:

- Remove the pump's AC power cord and rotating pole clamp. The power cord may remain attached if necessary.
- Place the pump into the open lockbox
- Push down the locking lever to secure the pump in place
- Reattach the AC power cord



Order Information

Medfusion® 3000 Series Lockbox 22-1000-51

THE DETAILS GIVEN IN THIS LEAFLET ARE CORRECT AT THE TIME OF GOING TO PRESS. THE COMPANY RESERVES THE RIGHT TO IMPROVE THE EQUIPMENT SHOWN.

Smiths Medical ASD, Inc.
St. Paul, MN 55112, USA
Phone: 1-214-618-0218
Toll-Free USA: 1-800-258-5361
www.smiths-medical.com

Smiths Medical is part of the global technology business Smiths Group plc.

Smiths Medical Canada Ltd.
Markham, Ontario, Canada, L3R 4Y8
Phone: 905-477-2000
Toll-Free: 1-800-387-4346

Customer and Clinical Services
U.S. 1-800-258-5361
Canada 1-800-387-4346
www.smiths-medical.com

Product(s) described may not be licensed or available for sale in Canada or other countries outside of the United States.

Medfusion and the Smiths Medical and Medfusion design marks are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2009 Smiths Medical family of companies. All rights reserved. 9/09 193814

smiths medical