

# cleo<sup>90</sup>

infusion set



Infusion sets play a key role in your diabetes management and your lifestyle.

You want to choose one that is compatible with your insulin pump, is easy to use, wears comfortably, and suits your personal needs.

*Simply*  
better.

## Easy insertion

The Cleo® 90 infusion set is a single-use 90° disposable infusion set. Its unique all-in-one housing design includes an inserter needle, cannula, and automatic needle retraction. Insertions are as easy as 1, 2, 3:



1 Remove the cap



2 Insert the cannula



3 Attach the tubing

## The advantages of the Cleo® 90 set are clear



### Enclosed needle safety

The enclosed needle automatically retracts up into the housing after insertion, helping to reduce accidental needlesticks.

### All-in-one design

The patented housing contains both a cannula inserter and needle retractor, providing you with maximum convenience and control.

### Hidden insertion needle


You control insertion speed — and only see the needle if you choose to.

### Variety of sizes

Available in 6- and 9-mm cannula lengths and 24-, 31- and 42-inch tubing lengths.

### Skin-friendly dressing

The small, transparent adhesive dressing breathes yet keeps water out — and stays in place with less tape. Its patented release tab makes removal easy, too.

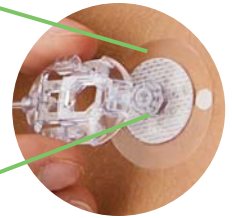

### Flex-Attach™ 360° site connection

The low-profile, rounded connector attaches to the site from any direction, for more control and flexibility.



### Standard Luer connection

Compatible with any open system insulin pump, though not for use with the MiniMed Paradigm® proprietary connection.

## Like no other set

Other features include:

- > Automatic needle retraction
- > Needle protection
- > Flex-Attach™ 360° site connection
- > Skin-friendly dressing that breathes yet keeps water out
- > Standard Luer connection  
(Not for use with the MiniMed Paradigm® proprietary connection.)



Only the Cleo® 90 infusion set is so simple and easy to use.

**For more information on the  
Cleo® 90 infusion set and to view  
a demonstration video, visit:  
[www.cleoinfusionsets.com](http://www.cleoinfusionsets.com)**

**800-826-9703**

**cleo<sup>90</sup>**  

---

**infusion set**

**smiths medical**

**Smiths Medical MD, Inc.**

St. Paul, MN 55112, USA

Phone: 1-800-426-2448

[www.smiths-medical.com](http://www.smiths-medical.com)

**Smiths Medical, part of the global  
technology business Smiths Group**

Cleo and the Smiths Medical design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2009 Smiths Medical family of companies. All rights reserved. 6/09 19744