

cleo⁹⁰

infusion set

With a Cleo™ 90 infusion set, you get:

- » Easy one-handed insertion and disconnection
- » Choice of 3 tubing lengths
- » Choice of 2 cannula lengths

No need to:

- » Touch a needle or needle guide
- » Attach the site and adhesive to the inserter
- » Manually insert or remove a needle



No more needles (well, almost...)

The Cleo™ 90 infusion set has an all-in-one needle inserter and retractor device that completely encloses the needle. Press the canister to insert the needle and you're done. The casing surrounds the inserter needle and automatically retracts it so you never have to touch or see a needle if you don't want to.

Easy disposal

Because the Cleo™ 90 infusion set is self contained, once the cannula is inserted, simply toss the container away in the trash. Other sets require you to use a sharps container or to close off the retracted needle manually before disposing of the set.

Secure connection

It takes only one hand to attach the 360° site connector tubing onto the site. As additional security, it will "click" into place so you know the site and the tubing are properly attached.

Cleo™ 90 infusion set ... Simply better.



The Cleo™ 90 infusion set has won two design awards:



Simply
better.

	Cleo™ 90	Inset®	Quick-Set®	Quick-Set® Paradigm
Insertion Device	X	X	X	X
Smallest Introducer	X			
Luer Connector	X	X	X	
Insertion Speed Control*	X			
Retraction Device	X			
Needle Protection	X			

*Users control the speed of insertion with the Cleo™ 90 infusion set. Others have only a spring-loaded, fast insertion.

Summarized Insertion Instructions

Cleo™ 90 set

1. Twist off the white cap.
2. Press the canister onto prepared site to insert. You control the insertion speed. A click indicates that the needle has retracted.
3. Attach the tubing from any direction.
4. Throw the used canister in the trash.

The Cleo™ 90 set is quick and convenient!

Inset®

1. Push up on the tab to remove top.
2. Unwind the tubing—be careful not to contaminate the Luer end.
3. Gently pull on the paper tab—unwind the paper clockwise to remove it from the adhesive.
4. Place the tubing in notch-hold.
5. Place your fingers in the indentions on the side of the infusion set.
6. Grasp the white plastic on the bottom of the set pull firmly to click into place.
7. Remove the needle cover.
8. Hold the tubing in place in notch.
9. Place the insertion set onto the prepared site.
10. Press firmly on the set sides at indentions—using equal pressure on each side. The needle will quickly insert.
11. Pull the insertion device straight up and out.
12. Replace the top to cover the needle.
13. Dispose of needle in secure sharps container.

Quick-Set®

1. Remove the infusion set from the package.
2. Replace the blue needle cover if removed to avoid contamination.
3. Place the infusion set into insertion device.
4. Remove the paper from adhesive (both sides).
5. Pull down on the bottom ring of the insertion device until it clicks.
6. Remove the blue needle cover.
7. Place the insertion device onto the prepared site.
8. Press both white buttons on either side of the insertion device at the same time. The needle will quickly insert.
9. Remove the insertion device.
10. Remove the blue section with the needle by pulling straight up.
11. Fold the narrow side of the blue section over the needle to cover it.
12. Do not the discard insertion device.
13. Dispose of needle in secure sharps container.

smiths medical

Smiths Medical MD, Inc.
St. Paul, MN 55112, USA
www.smiths-medical.com
Smiths Medical - a part of Smiths Group plc

www.cleoinfusionsets.com 1-800-826-9703

cleo⁹⁰
infusion set

Cleo and the Smiths Medical design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark office, and certain other countries. All other names and marks mentioned are the trade names, trademarks or service marks of their respective owners. ©2006 Smiths Medical family of companies. All rights reserved. 11/06 19748