


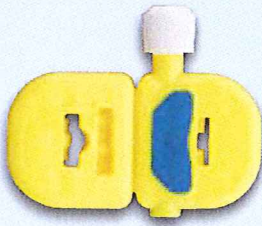

Comparing EpiFuse™ Catheter Connector with Current Smiths Medical Connectors

	EpiFuse™ Catheter Connector	Tuohy Borst Connector	Locking Connector
			
Definitive Closure Click	✓	✗	✓
Reversible	✓	✓	✗
Yellow for "Epidural"	✓	✗	✗
Flat Profile	✓	✗	✓
One Hand Closure	✓	✗	✗
Tamper Resistant	✓	✗	✓

Smiths Medical International Ltd

Hythe, Kent CT21 6JL UK
Tel: +44 (0)1303 260551 Fax: +44 (0)1303 266761
www.smiths-medical.com




Smiths Medical is part of the global technology
business Smiths Group plc.

The Smiths Medical and Portex design marks and EpiFuse are trademarks of the Smiths Medical family of companies. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2010 Smiths Medical family of companies. All rights reserved. 2/10 LIT/PM 3063



Comparing EpiFuse™ Catheter Connector with Current Smiths Medical Connectors

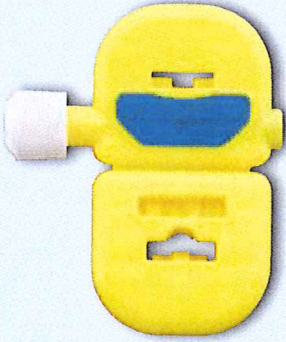smiths medical
bringing technology to life

EpiFuse™ Catheter Connector	Tuohy Borst Connector	Locking Connector
		
<p>Description</p> <ul style="list-style-type: none"> • Consists of two halves joined by a living hinge • Catheter is inserted into the hole at the base of the connector • Catheter is retained when the two halves of the connector are folded and locked together • Definitive stopping point to indicate the catheter is secure • To release the catheter, a Luer slip device is inserted into the release aperture; the two halves of the connector will separate allowing for removal of the catheter 	<ul style="list-style-type: none"> • Consists of a top and bottom half that are threaded together • Catheter is inserted into the hole in the bottom half of the connector • As the halves are twisted together, they tighten to grip the catheter • To release the catheter the two halves are unscrewed 	<ul style="list-style-type: none"> • Consists of two halves joined by a living hinge • Catheter is inserted into the hole at the base of the connector • Catheter is retained when the two halves of the connector are folded and locked together • Definitive stopping point to indicate the catheter is secure • Once the connector is closed and locked, it cannot be opened to release the catheter

Smiths Medical International Ltd

Hythe, Kent CT21 6JL UK

Tel: +44 (0)1303 260551 Fax: +44 (0)1303 266761

www.smiths-medical.com

Smiths Medical is part of the global technology business Smiths Group plc.

The Smiths Medical and Portex design marks and EpiFuse are trademarks of the Smiths Medical family of companies. The symbol @ indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2010 Smiths Medical family of companies. All rights reserved. Z/10 LIT/PM 3062

TM

PORTEX