

# PressureEasy®


## Disposable Endotracheal Tube Cuff Pressure Controller

- Monitors endotracheal cuff pressure level between 20–30cm H<sub>2</sub>O
- Guards against aspiration and tracheal damage
- Pressure feedback line designed to eliminate cuff leaks at peak inspiratory pressure




# PressureEasy®


## Disposable Endotracheal Tube Cuff Pressure Controller

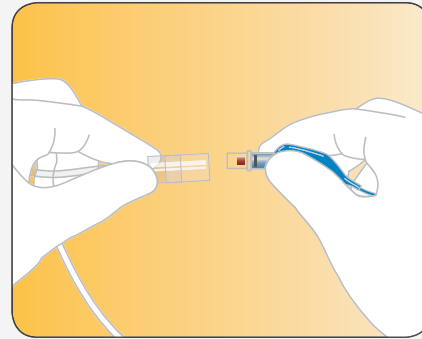


**Attach PressureEasy®** luer slip connector to the pilot balloon port on the ET tube. Ensure full insertion to prevent possible leaks.

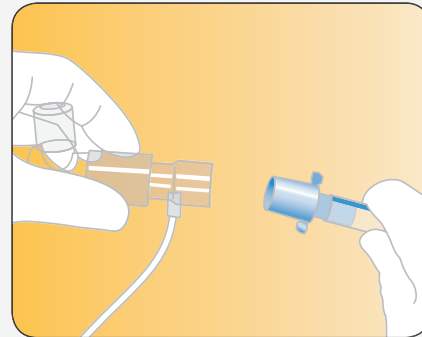

**Attach high pressure adaptor** between the ET tube and ventilator circuit if needed.

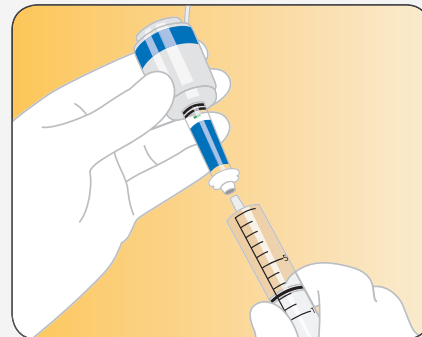

**Attach syringe to stem** of the PressureEasy® Cuff Pressure Controller device.

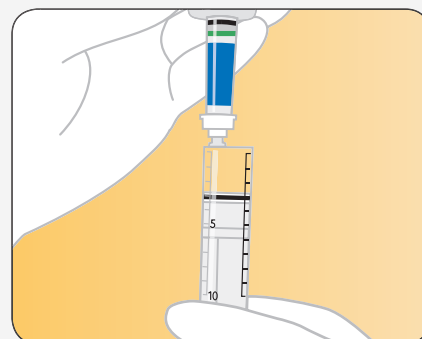

**Inflate tracheal cuff** until the green indicator is visible in the pressure window, indicating pressure between 20–30cm H<sub>2</sub>O.

### PressureEasy® Cuff-Pressure Controller

CATALOG NO.	DESCRIPTION	UNITS/CASE
292004	PressureEasy® Cuff Pressure Controller*	10

\*Single patient use only

# smiths

## Smiths Medical

2231 Rutherford Road  
 Carlsbad, California 92008, USA  
 T: 800 848 1757 F: 800 621 2590  
 www.smiths-medical.com

Caution: Federal law [U.S.A.] restricts this device to sale by or on the order of a physician.

PressureEasy and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates it is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are trade names, trademarks, or service marks of their respective owners.

©2006 Smiths Medical family of companies. All rights reserved.

RE209 2006.7