

# GRIPPER<sup>®</sup> Micro

Blunt Cannula, Non-Coring Safety Needle

## Insertion



## Access a port

Prepare the site according to facility protocol. Holding on to the inserter as shown, insert the needle into the port at a 90° angle.



Finger on tip



Press tab in




Lift safety arm

## Remove the needle

- From the back of the inserter, place fingers on each side of the inserter's base to stabilize it.
- With the other hand, place a finger on the tip of the inserter's safety arm.
- **Press the tab in** and lift the safety arm straight back until the needle **CLICKS** into the locked position.

✓ **The sharp is now removed.**

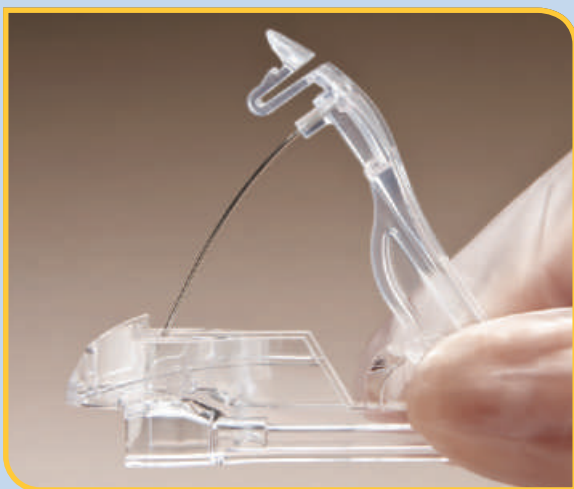

## Dispose in sharps container

- Dispose of the inserter in a sharps container.
- Apply a semi-permeable dressing over the infusion site, ensuring that a minimum 4cm area surrounding the site is covered.

## Removal



## Remove infusion site

Place fingers on each side of the infusion site. Stabilize the port with the other hand. Lift the infusion site straight up and discard per facility protocol.

## GRIPPER<sup>®</sup> Micro Needle Sizes

Available with a non y-site and Luer-activated needleless y-site (NAC)

Gauge	Length	Reorder numbers	
		Non y-site	NAC
19	3/4"	21-3260	21-3280
19	1"	21-3261	21-3281
19	1 1/4"	21-3262	21-3282
20	3/4"	21-3256	21-3276
20	1"	21-3257	21-3277
20	1 1/4"	21-3258	21-3278
22	3/4"	21-3251	21-3271
22	1"	21-3252	21-3272
22	1 1/4"	21-3253	21-3273

Smiths Medical ASD, Inc.  
St. Paul, MN 55112, USA  
Phone: 1-214-618-0218  
Toll-Free USA: 1-800-258-5361  
www.smiths-medical.com

Smiths Medical Canada Ltd.  
Markham, Ontario, Canada, L3R 4Y8  
Phone: 905-477-2000  
Toll-Free: 1-800-387-4346



EC Authorized Representative  
Smiths Medical International Ltd.  
WD24 4LG, UK  
Phone: +44 (0) 1923 246434

Customer and Clinical Services  
U.S. 1-800-258-5361  
Canada 1-800-387-4346  
www.smiths-medical.com

**Deltec**

Product(s) described may not be licensed or available for sale in Canada or other countries outside of the United States.  
GRIPPER and the Smiths Medical and Deltec design marks are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2010 Smiths Medical. All rights reserved. 9/10 VA19389

smiths medical