

FOR MORE INFORMATION VISIT  
[WWW.SMITHS-MEDICAL.COM](http://WWW.SMITHS-MEDICAL.COM)



## CADD®-SOLIS SYSTEM WITH PROGRAMMED INTERMITTENT BOLUS (PIB)

### A versatile system from the leader in ambulatory infusion pumps

- For over 35 years, our CADD®, Medfusion®, and PharmGuard® software infusion products have supported our passion for having a positive impact on peoples' lives and helping our customers make lives better.

#### Celebrating 5 Years in the Market – CADD®-Solis System with PIB

In 2014, we introduced the CADD®-Solis system with PIB as your solution to pain management for labor analgesia and post-op pain management. The CADD®-Solis pump contains the following epidural delivery modes:

- Continuous epidural infusion (CEI)
- Patient-controlled epidural analgesia (PCEA)
- Programmed intermittent bolus (PIB)

The pump also provides a lockout protocol that prioritizes either the delivery of a PCA dose or a programmed intermittent bolus and limits the total amount of drug which can be delivered within a defined time frame.\*





\*Hourly limit does not include clinician boluses

smiths medical

# Programming Flexibility

With the CADD<sup>®</sup>-Solis system, you have the flexibility to delay a PIB following a PCEA dose based on the PIB interval or PCEA dose lock out interval.

## Intermittent Bolus Examples

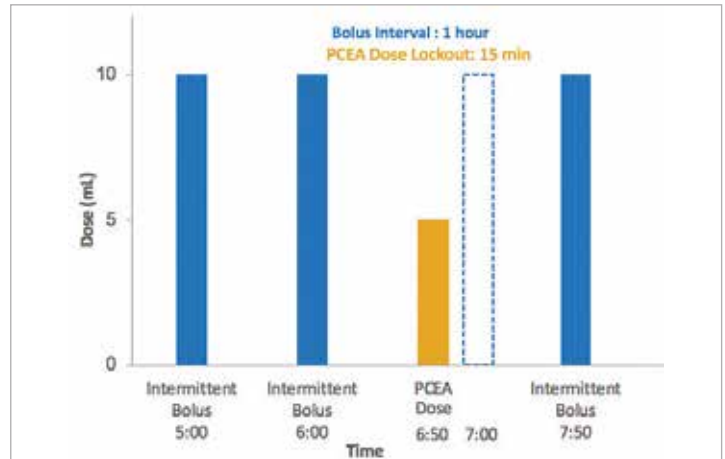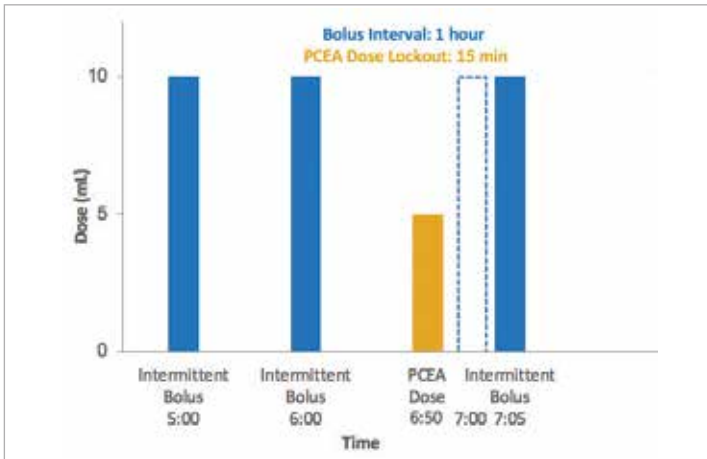

**Bolus Interval Type Setting = PCEA Dose Lockout** – Intermittent boluses will deliver every hour unless the time between a delivered PCEA Dose and the next scheduled PIB is less than the PCEA Dose Lockout time

**Bolus Interval Type Setting = PIB Bolus Interval** – Intermittent boluses will deliver every hour unless a PCEA Dose is delivered



## CADD<sup>®</sup>-Solis Ambulatory Infusion System for Pain Management

- Single platform for IV PCA, Epidural, PNB, and PIB therapies
- Customizable color-coded screens to differentiate therapies
- Customizable programming limits
- Variety of point of care reports
- Allows wireless data collection, library updates, and firmware upgrades

For more information visit our website at [www.smiths-medical.com](http://www.smiths-medical.com)

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES.

**Smiths Medical ASD, Inc.**  
6000 Nathan Lane North  
Minneapolis, MN 55442, USA  
Tel: 1-614-210-7300  
Toll-Free USA: 1-800-258-5361  
[www.smiths-medical.com](http://www.smiths-medical.com)

Find your local contact information at: [www.smiths-medical.com/customer-support](http://www.smiths-medical.com/customer-support)

MMSPCA-0584

**Rx  
ONLY**

smiths medical

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. CADD, Medfusion, PharmGuard and the Smiths Medical design mark are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. ©2019 Smiths Medical. All rights reserved. IN193308EN-052019