



For use with  
the CADD®-Solis VIP  
ambulatory  
infusion system

# Protocol Details Programming Record

## CADD™-Solis Medication Safety Software – Administrator

### Taper/TPN Therapy

INSTITUTION

---

DATE

---

Library General Settings

Setup Therapies, Qualifiers, and Drugs

Specify Protocols

### Taper

LIBRARY

THERAPY

QUALIFIER

DRUG NAME

UNITS (Milliliters, Milligrams, Micrograms)

CONCENTRATION

Delivery 1

Delivery 2

Alarms

Security

Display/Sound

#### Plateau Rate

mL /hr

Maximum

Initial Value:

(calculated)

#### Infusion Volume

mL

- Display  
 Allow Edit  
 Require Comment

Hard Maximum:

Soft Maximum:

Initial Value:

Soft Minimum:

Hard Minimum:

#### Infusion Duration

- Display  
 Allow Edit

Initial Value:

#### Taper-Up Period

- Display  
 Allow Edit

Initial Value:

#### Taper-Down Period

- Display  
 Allow Edit

Initial Value:

#### Requires Verification

- Drug  
 Second Nurse

Active – Protocol Available to Send to Pump

Save

Cancel

Help

**Check Active box when the protocol is ready to be sent to a pump**

**Save** allows you to save the protocol details tabs without making the protocol available to send to a pump.

**Cancel** allows you to return to selecting a therapy, qualifier, and drug and will delete any changes made.

**Help** provides additional information in creating the protocol details section.

AUTHORIZED BY

DATE

ENTERED BY

DATE

VERIFIED BY

DATE

Delivery 1

Delivery 2

Alarms

Security

Display/Sound

**Reservoir Reset Volume**

- Display
- Allow Edit

mL (0-9999)

**KVO Rate**

- Display
- Allow Edit

mL/hr

Save

Cancel

Help

Delivery 1

Delivery 2

Alarms

Security

Display/Sound

Pump Stopped Alarm:

- Informational
- High Priority

**Reservoir Low**

Trip Point:

mL (1-999)

Alarm Type:

- Insistent and One Time Only
- Non-insistent and Repeating

Reservoir Empty Alarm Type:

- One Time Only
- Repeat Until Removed

**Air Detector**

Enable:

- Off
- On

Sensitivity:

- High
- Low

Upstream Sensor:

- Off
- On

Downstream Sensor Sensitivity:

- High
- Low

**Preventative Maintenance**

Reminder:

- Off
- On

Interval:

months (1-24)

Save

Cancel

Help

## Codes

Keypad Code:

 (001-899)

Clinician Code:

 (001-899)

Admin Code:

 (001-899)

## Patient Permissions

Priming Requires Security:

 Off  
 On

Delayed Start Requires Security:

 Off  
 On

Manual Programming Security:

 Admin Code  
 Clinician Code

Keypad Security:

 Code Only  
 Code or Key

Save

Cancel

Help

Backlight Intensity:

 (1-10)

Alarm Volume:

 Low  
 Medium  
 High

Sound Theme:

 Standard  
 Intense  
 Distinctive

Key Beep:

 Off  On

Date Format:

 Month/Day/Year  
 Day/Month/Year  
 Year/Month/Day

Time Format:

 12:00:00 AM/PM  
 12.00.00 AM/PM  
 24:00:00  
 24.00.00

Numeric Format:

 1,234.56  
 1.234,56  
 1 234.56  
 1 234,56

Support Message:

  

Save

Cancel

Help

## Delivery 1 Tab

### Plateau Rate Maximum

Allows you to program the maximum allowable rate at which the medication may infuse during the plateau portion of the delivery. The initial value is calculated and displayed based on the Infusion Volume, Infusion Duration, and any Taper Periods programmed.

### Display/Allow Edit/Require Comment

*For use with Single Protocol PC programming only.*

Optional settings to determine whether the Delivery Parameter will be displayed, eligible for edit, and/or a comment entry will be required by the clinician should the initial setting be programmed outside of a soft range limit.

### Infusion Volume

Allows you to program the total volume of fluid to be delivered over the duration of the Taper infusion along with Hard and Soft titration range limits for this value. If you enter an Infusion Volume that causes the Plateau Rate to exceed the maximum allowable rate, the Infusion Duration will automatically lengthen to accommodate the volume.

### Infusion Duration

This is the time required to deliver the Infusion Volume. The Infusion Duration includes any Taper Up or Taper Down Periods, if programmed.

### Taper – Up Period

Allows you to program the duration for the taper up period.

### Taper – Down Period

Allows you to program the duration for the taper down period.

### Requires Verification

*For use with Single Protocol PC programming only.*

Require drug verification via Drug ID entry or barcode scan and/or require second nurse User ID and Password entry, when selected.

## Delivery 2 Tab

### Display/Allow Edit

*For use with Single Protocol PC programming only.*

Optional settings to determine whether the Delivery Parameter will be displayed and/or the initial setting will be eligible for edit by the clinician using Single Protocol programming with the PC.

### Reservoir Reset Volume

Allows you to program the volume of fluid in the reservoir. When you change the reservoir and reset the Reservoir Volume, the volume resets to the value programmed in this screen.

### KVO Rate

The KVO or "Keep Vein Open" rate is optional. It allows the delivery of a minimal amount of drug to help maintain catheter patency. If a Delayed Start is programmed, the KVO rate is active until the infusion begins. It is also active after the Infusion Volume is delivered if the Reservoir Volume is programmed greater than the Infusion Volume.

## Alarms Tab

### Pump Stopped Alarm

Allows you to select either a High Priority or Informational alarm to remind the clinician that the pump is stopped. The alarm will recur every five minutes while the pump is stopped.

**High Priority:** the user will be required to acknowledge the alarm.

**Informational:** the user is not required to acknowledge the alarm.

### Reservoir Low Trip Point

Allows you to program the pump to alarm when the reservoir volume reaches a specified level.

### Reservoir Low Alarm Type

Programmable if the value for the Trip Point is greater than zero. Allows you to select either a Medium Priority or Low Priority alarm to inform the clinician that the Reservoir Volume is low.

**Insistent and One Time Only:** Medium Priority – the user will be required to acknowledge the alarm, it will not reoccur once it has been acknowledged.

**Non-Insistent and Repeating:** Low Priority – the user is not required to acknowledge the alarm, it will repeat at the 75%, 50%, and 25% marks of the amount determined as the Trip Point.

### Reservoir Empty Alarm Type

Allows you to select between two types of High-Priority alarms when the reservoir is empty.

**One Time Only:** the user will be required to acknowledge the alarm, it will not reoccur once it has been acknowledged.

**Repeat Until Removed:** the user will be required to acknowledge the alarm, it will repeat every 5 minutes until the Reservoir Volume has been set to a new value, the reservoir has been removed, or the pump is powered down.

### Air Detector Enable

Allows you to control whether the Air Detector is turned on or off. This is a High-Priority alarm.

### Air Detector Sensitivity

Programmable only if the Air Detector is set to "On". Allows you to customize how sensitive you want the pump to be to the air bubbles in the tubing.

### Upstream Sensor

Allows you to control whether the Upstream Sensor is turned on or off. If this is programmed to "On", a High-Priority alarm will sound if an occlusion is detected between the pump and the fluid container. It is only detected when administration sets and bags are used. The pump will not test for an upstream occlusion with medication cassette reservoirs even when turned "On".

### Downstream Sensor Sensitivity

Allows you to specify a High or Low degree of response to pressure buildup in the downstream tubing. **Note:** *Downstream Sensor is always activated. This is a High-Priority alarm.*

### Preventative Maintenance Reminder

Allows you the option to program the pump to sound a Medium-Priority alarm when it is time for preventative maintenance. The message will begin appearing on the date programmed and during every power up until the date is reset.

### Preventative Maintenance Interval

Allows you to schedule the amount of time that should pass before a PM Reminder displays.

## Security Tab

### Codes

Allows you to customize the Keypad Code, Clinician Code, and/or the Administrator Code.

### Manual Programming Security

Allows you to choose which security code will allow access to the Manual Program protocol.

### Keypad Security

Allows you to choose whether you want the clinician to have the option to choose between using a code or the key to unlock the keypad, or whether the clinician should be restricted to using the code only.

### Patient Permissions

**Priming Requires Security** - Setting this value to "Off" allows patient/caregiver to prime the tubing on the pump without having to enter a security code.

**Delayed Start Requires Security** - Setting this value to "Off" allows patient/caregiver to set Delayed Starts without having to enter a security code.

## Display/Sound Tab

### Backlight Intensity

Allows you to adjust the backlight intensity within the range of 1 to 10. **NOTE:** *Increasing the Backlight Intensity will shorten the battery life.*

### Alarm Volume

Allows you to determine the volume of the alarms in the protocol. **NOTE:** *High alarm volumes will shorten battery life.*

### Sound Theme

Allows you to choose between 3 different sound themes for the alarms and beeps on the pump.

### Key Beep

Allows you to turn on or off the audible beep which accompanies each key press.

### Date Format

Allows you to choose the format for how the date is displayed.

### Time Format

Allows you to choose either a 12-hr or 24-hr clock format, and to set the character used for indicating hours, minutes, and seconds separators.

### Numeric Format

Allows you to choose the character or space used to indicate the decimal point and thousands units separators.

### Support Message

Allows you to type a customized support message viewable on the pump at the end of an Alarm Help Screens sequence.

**NOTE:** *The character length is restricted.*

### Smiths Medical ASD, Inc.

St. Paul, MN 55112, USA  
Phone: 1-214-618-0218  
Toll-Free USA: 1-800-258-5361



### EC Authorized Representative

Smiths Medical International Ltd.  
TN25 4BF, UK  
Phone: +44 (0) 1233 722100

**Rx  
ONLY**

www.smiths-medical.com Find your local contact information at [www.smiths-medical.com/customer-support](http://www.smiths-medical.com/customer-support)

Smiths Medical is part of the global technology business Smiths Group plc. MPAUCA-1042 Product(s) described may not be licensed or available for sale in all countries. Please see the Instructions for Use/Operator's Manual for a complete listing of indications, contraindications, warnings and precautions. CADD-Solis, CADD and the Smiths Medical and CADD design marks are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2013 Smiths Medical. All rights reserved. IN193570EN-062013

smiths medical