

CADD®-Solis VIP Ambulatory Infusion System

Alarm Help Screens



When certain alarms occur, additional information may be displayed on the screen. The help screens describe what you can do to try to solve the problem that is causing the alarm.

STEPS:






1. When an alarm occurs, press **SILENCE** to quiet the alarm.
2. If help screens are available for the alarm, "Help" appears above the right soft key. To view the help screens, select **HELP**.
3. Follow the applicable instructions provided on the help screen. To page through all of the available help screens, press **NEXT** repeatedly. Press **ACKNOWLEDGE** at any time to exit Help.
4. "Retry Help" appears when no additional help steps are available. To page through the help screens again, press **RETRY HELP**. The alarm screen reappears as in Step 2 above. Repeat steps 2 and 3. When the alarm clears, the help screen will no longer be displayed.

The Following Alarms Have Associated Help Screens. Refer to Steps Above to Navigate the Help Screens.

CANNOT START PUMP WITHOUT A LATCHED CASSETTE.



UPSTREAM OCCLUSION. CLEAR OCCLUSION BETWEEN PUMP AND RESERVOIR.


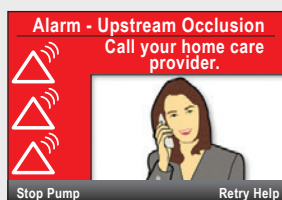
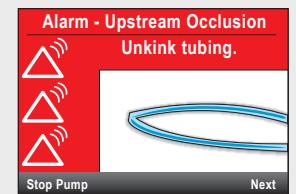
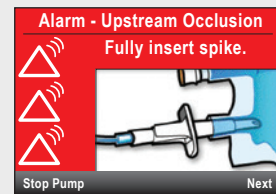
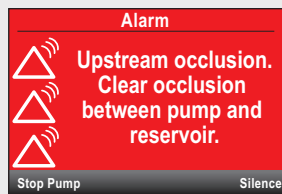
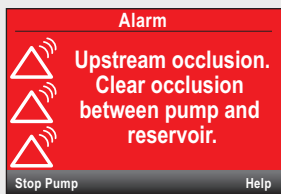


CADD®-Solis VIP Ambulatory Infusion System


Alarm Help Screens

The Following Alarms Have Associated Help Screens.
Refer to Steps Above to Navigate the Help Screens.



DOWNSTREAM OCCLUSION. CLEAR OCCLUSION BETWEEN PUMP AND PATIENT.

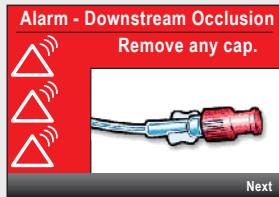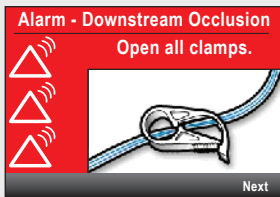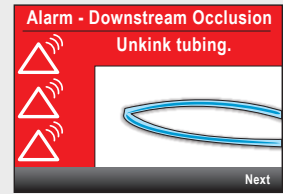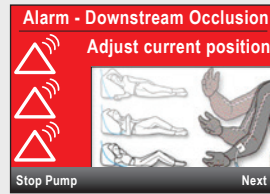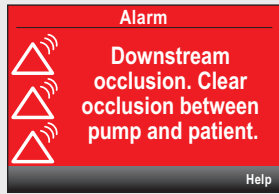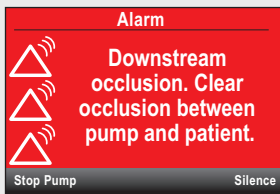

CANNOT START PUMP WITHOUT A LATCHED AND LOCKED CASSETTE. (PCA ONLY)

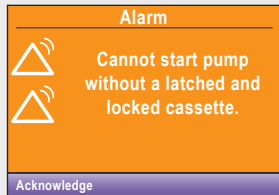

LOCK CASSETTE TO START PUMP. (PCA ONLY)

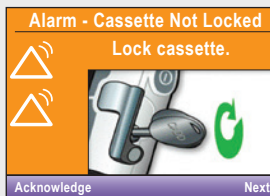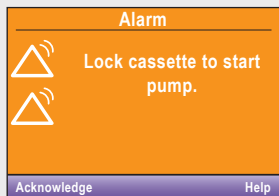

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical ASD, Inc.
6000 Nathan Lane North
Minneapolis, MN, 55442, USA
Tel: 1-614-210-7300
Toll-Free USA 1-800-258-5361
www.smiths-medical.com

Find your local contact information at: www.smiths-medical.com/customer-support

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. CADD and the Smiths Medical design mark are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. ©2017 Smiths Medical. All rights reserved. IN193583EN-082017

MPAUC-1617

Rx
ONLY

smiths medical