





CADD[®]-Solis VIP Ambulatory Infusion Pump Taper Mode*


PROGRAMMING A NEW PATIENT

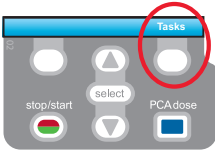

1. To start programming, select **Tasks**.

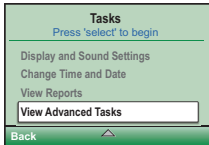


2. Press to highlight **View Advanced Tasks** + **select**.

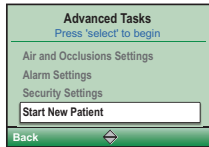


3. Using the key, highlight **Start New Patient** + **select**.
Then select **Continue**.

4. Enter the 3-digit security code using or keys + **select**.

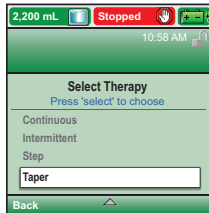

5. Using the or keys, highlight **Taper** + **select**.
Highlight **[Manual Progr]** or protocol qualifier if existing + **select**.
If protocol, highlight drug name + **select**.

6. Select **Yes** to confirm selection. Select **Review** to **Review Pump Settings**.

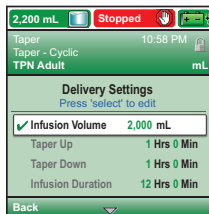

7. Next, adjust all of the settings. Highlight a setting, press **select** then or then select **Save**.

Reminder

Infusion Volume: total volume of fluid to be delivered
Reservoir Volume: volume of fluid contained in the bag

8. Confirm values by selecting **Accept Value**, the symbol appears on the left + **Next**.

9. Remove the blue clip and attach the tubing. The pump will show the next steps to follow until pump start.

10. Start the pump.



*Taper Mode: commonly used for parenteral nutrition



Quick Change of Disposable

Same Infusion Volume, Same Programming



1. Connect the CADD® administration set to the bag.
Without removing the blue clip on the cassette, prime the tubing manually until all air is removed. Or follow instructions below to prime using the pump.
2. Turn on the pump.
3. Remove the blue clip and attach the new disposable to the pump.
4. The pump displays the question **Reset reservoir volume to XX mL?**
Select **YES** to confirm.
5. The screen displays **Prime tubing?**
Select **YES** to prime using the pump. Select **No** if tubing was pre-primed.
6. Enter the 3-digit security code. Press  or  + **select** if needed.
7. Start the pump.

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical International Ltd
1500 Eureka Park, Lower Pemberton
Ashford Kent, TN25 4BF
Tel: +44 (0)845 850 0445
www.smiths-medical.com

Smiths Medical ASD, Inc.
Minneapolis, MN 55442, USA
Phone : +1-614-210-7300

EC Authorized Representative
Smiths Medical International Ltd.
TN25 4BF, UK
Phone: +44 (0) 1233 722100

MPAUC-1640

smiths medical

Find your local contact information at: www.smiths-medical.com/customer-support

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings, and precautions. CADD and the Smiths Medical design mark are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. Product referenced is CE-marked. ©2017 Smiths Medical. All rights reserved. IN193588GB-102017