


## Previewing Display and Sound Settings on a CADD®-Solis Pump

### Display and Sound Process


Display and sound settings are customizable using the medication safety software. Most of these settings can be previewed on a pump to help facilitate configuration decisions. The Admin code (highest security level) is required to access the display and sound settings. If you do not have access to this code, please contact your library administrator. If the pump is in the factory default mode (i.e. no library has been loaded to the pump yet), then the Admin code will be the default factory code.

**Note:** Screen shots below may differ from what you will see on your pump, depending on your pump version and library settings.


1. Power on the pump. Select **No** to the question, "Do you want to start a new patient?" Then select **Tasks**.

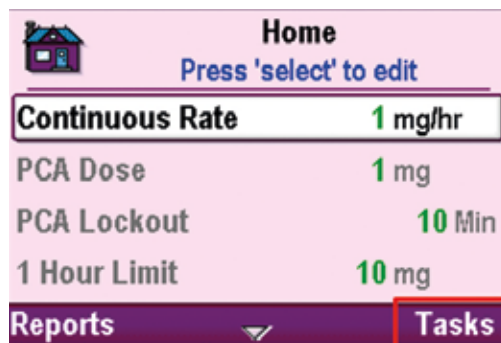

2. Scroll down and select **Adjust Admin Settings**. Enter the Admin code to continue.

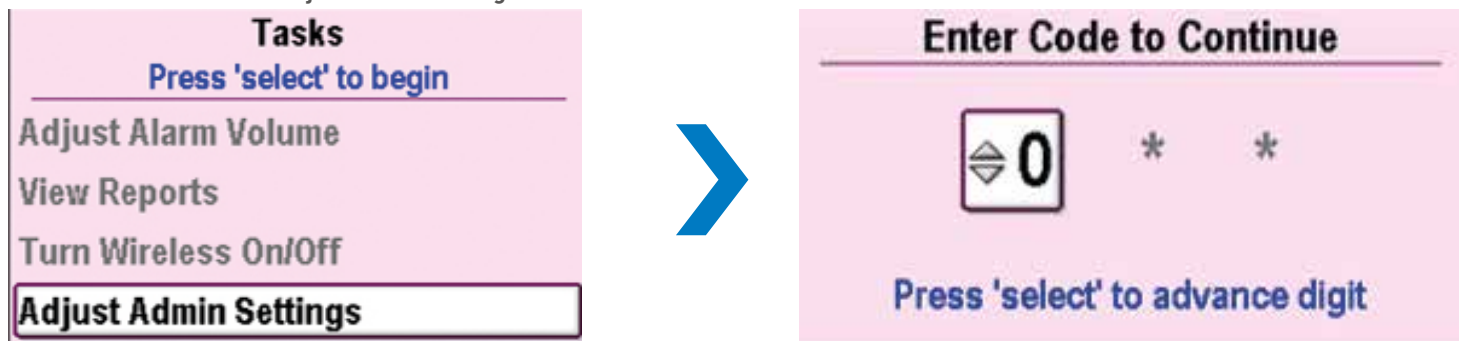

3. Scroll down and select **Display and Sound**.

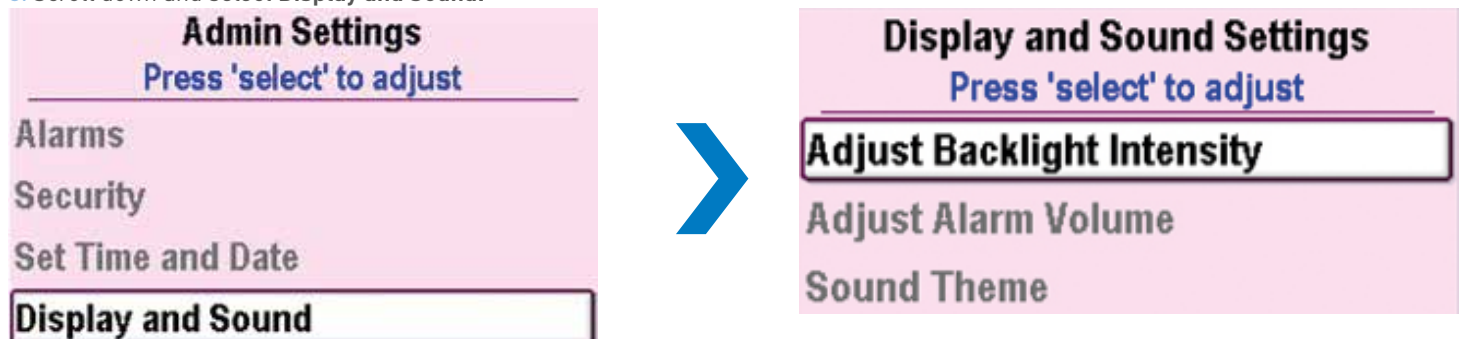


## BACKLIGHT INTENSITY

1. The Backlight Intensity ranges from 1-10. From the **Display and Sound Settings** menu, scroll to select **Adjust Backlight Intensity**.
2. Scroll up and down to preview the backlight intensity at the different settings.
3. To exit, select **Don't Save**




## ALARM VOLUME

1. Three alarm volume options are available. From the **Display and Sound Settings** menu, scroll to select **Adjust Alarm Volume**.
2. Select each alarm volume option to preview.
3. To exit, select **Don't Save**

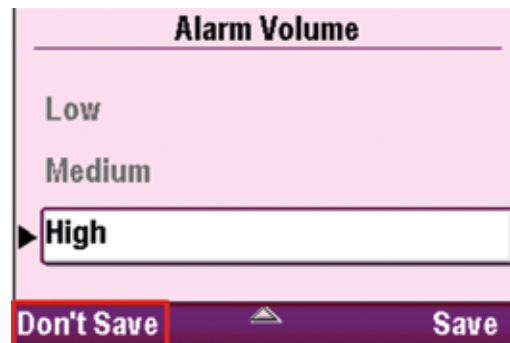

## SOUND THEME

1. Three sound theme options are available. From the **Display and Sound Settings** menu, scroll down and select **Sound Theme**.
2. In the **Sound Theme** screen, press select to preview a theme. In the **Sound Theme Preview** screen, press **Select** to listen to each alarm type within the theme.
3. Repeat step 2 to preview the remaining sound themes.
4. To exit, select **Back** and then **Don't Save**

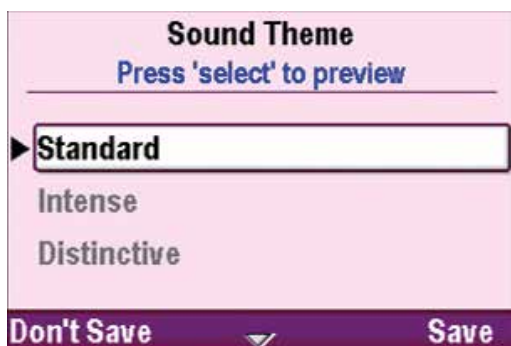

For more information visit our website at [www.smiths-medical.com](http://www.smiths-medical.com)

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

**Smiths Medical ASD, Inc.**  
6000 Nathan Lane North  
Minneapolis, MN 55442, USA  
Tel: 1-614-210-7300  
Toll-Free USA: 1-800-258-5361  
[www.smiths-medical.com](http://www.smiths-medical.com)

MMSPCA-0336

**Rx  
ONLY**

Find your local contact information at: [www.smiths-medical.com/customer-support/contact-us](http://www.smiths-medical.com/customer-support/contact-us)

**smiths medical**

Smiths Medical is part of the global technology business Smiths Group plc. Product(s) described may not be licensed or available for sale in all countries. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Medfusion and the Smiths Medical design mark are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2018 Smiths Medical. All rights reserved. IN193599EN-082018