


## Portex® Regional Anaesthesia System with NRFit® Connector

### Infusion Adapter Connection Instructions

The CADD™ Infusion Adapter with male NRFit™ connector and female Luer is intended to be used by clinicians to convert administration sets with Luer connectors into permanently dedicated lines for use only with NRFit™ connectors. The NRFit™ connector with non-Luer taper is intended for the injection or infusion of regional anaesthetics or narcotics and may help reduce the risk of misconnection or misinjection.

**IMPORTANT:** The infusion adapter must be connected to the infusion line first for proper function of the device.


#### Infusion Adapter Components

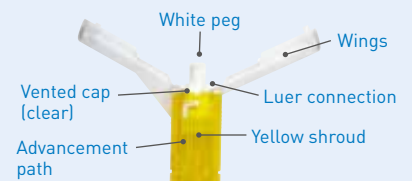

1. Prepare and prime the infusion set under hospital protocol with prescribed medications.
2. Grasp the adapter by the yellow shroud, placing thumb and forefinger on the finger grips. Attach the infusion set to the adapter by turning the collar of the set's male Luer clockwise onto the adapter.



3. Continue turning the infusion set clockwise until the white pegs beneath the shroud are aligned with the advancement path.

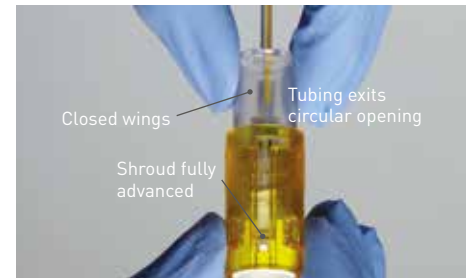

4. Close the wings around the male Luer of the infusion set. Make sure the infusion set tubing runs through the circular opening at the end of the closed wings.
5. Push the closed wings forward through the shroud until the white pegs snap into place at the end of the path, exposing the white NRFit® connector with the vented cap (this locks the infusion adapter onto the infusion set, dedicating it for use with NRFit® regional anaesthesia components).



6. Prime the adapter.
7. When you're ready to begin the infusion, remove the vented cap from the NRFit® connector side.




8. Connect to the NRFit® regional anaesthesia connection with an appropriate epidural catheter, or with an epidural flat filter with NRFit® regional anaesthesia connector.  
**Do not overtighten.**



9. Attach the labels. Recommended placement of the labels (see the figure above) is distal and proximal to the NRFit® neuraxial connections.

# Infusion Adapter

Allows connection of a pump Luer to a NRFit® connector on a Portex® regional anaesthesia product.

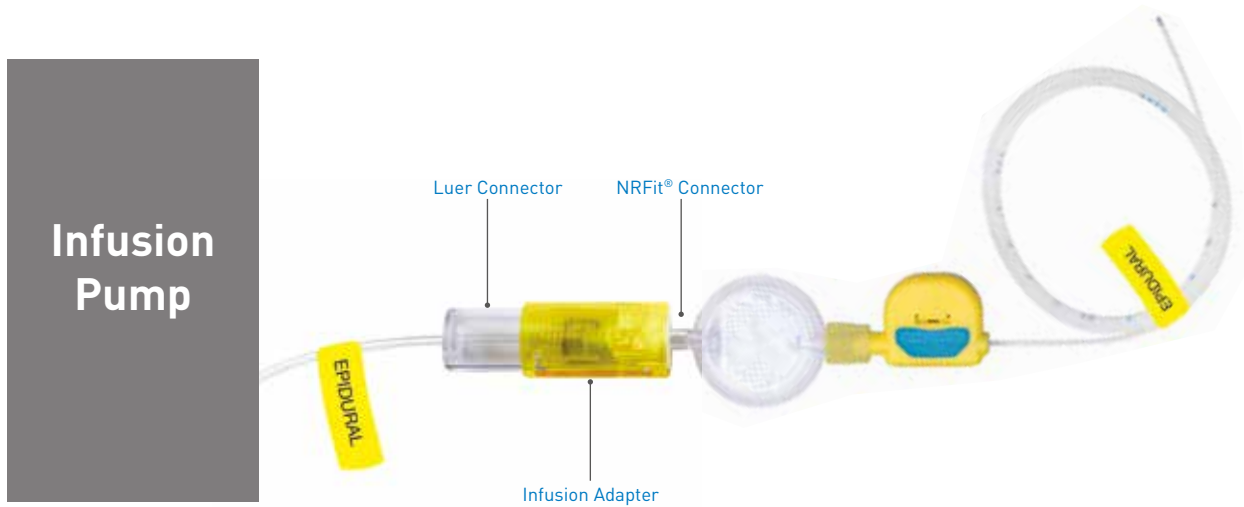

## Ordering Information

Infusion Adapter		
Description	Product Code	Unit
Infusion adapter to lock a male Luer on the a female NRFit® connector	24-1001-64	10/box

## Contraindications

- The infusion adapter is not indicated for use with Luer slip syringes or Luer lock syringes.
- The infusion adapter is not indicated for use with Luer catheter connectors or Luer epidural filters.
- Follow known contraindications for regional anaesthesia procedures as stated in standard textbooks.

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN ALL GEOGRAPHIES

**Smiths Medical ASD, Inc.**  
6000 Nathan Lane North  
Minneapolis, MN, 55442, USA  
Tel: 1-614-210-7300  
Toll-Free USA 1-800-258-5361  
[www.smiths-medical.com](http://www.smiths-medical.com)

Find your local contact information at: [www.smiths-medical.com/customer-support](http://www.smiths-medical.com/customer-support)

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Portex, CADD and the Smiths Medical design mark are trademarks of Smiths Medical. NRFit is a registered trademark of GEDSA used with their permission. The symbol® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. ©2019 Smiths Medical. All rights reserved. IS0045.GB.GB.Rev.A.1219



smiths medical