

HemoDraw® Plus Closed Blood Sampling System - Single Port

Quick Blood Sampling Operation Guide



1 Prepare the system to obtain blood samples

- Verify the reservoir is locked in the 0 mL position
- Locate and cleanse sampling site




- Activate HemoDraw® reservoir slowly and lock into 5 mL position



2 Obtain blood samples

- Connect a Luer lock syringe or sampling device to sampling site

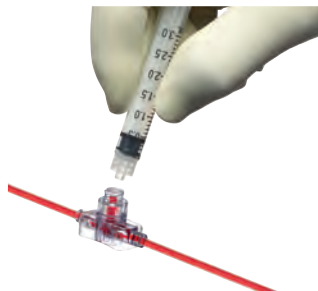

- Gently withdraw blood sample



- Vertically disconnect Luer lock syringe or sampling device from sampling site

3 Re-infuse blood and flush system

- Unlock reservoir from 5 mL position and slowly re-infuse the patient's blood. Return and lock reservoir in the 0 mL position for monitoring



- Activate flush device until blood is cleared from line. Short flushes may clear line better



- Cleanse sampling site

This Operation Guide demonstrates the main steps for obtaining a blood sample using the HemoDraw® Plus Closed Blood Sampling System. Prior to using this system, you must receive full training and read the Instructions For Use for detailed instructions, warnings and cautions associated with the use of this device.

Blood should be drawn and infused slowly to reduce the risk of degassing. Draw and infusion rates should not exceed 0.5 mL per second (≤ 0.5 mL/s).

For more information visit our website at www.smiths-medical.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

MDBLCA-1069

Smiths Healthcare Manufacturing SA de CV
Carretera Miguel Alemán Km 21.7
Parque Industrial Monterrey
Apodaca NL, CP 66603 Mexico
www.smiths-medical.com

Smiths Medical ASD, Inc.
6000 Nathan Lane North
Minneapolis, MN 55442, USA
Tel: 1-614-210-7300
Toll-Free USA: 1-800-258-5361

**Rx
ONLY**

Find your local contact information at: www.smiths-medical.com/customer-support

smiths medical

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. HemoDraw, Medex and the Smiths Medical design mark are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. ©2017 Smiths Medical. All rights reserved. MX195054GB-102017