

# smiths medical

## portex®



### PressureEasy® Cuff Pressure Controller



- Monitors tracheal cuff pressure level between 20–30cm H<sub>2</sub>O
- Guards against aspiration and tracheal damage
- Pressure feedback line designed to eliminate cuff leaks at peak inspiratory pressure

# PRESSUREEASY<sup>®</sup> CUFF PRESSURE CONTROLLER

**Luer slip connector** easily attaches to pilot balloon port of tracheal tube.



**High pressure adaptor** adjusts cuff pressure during positive pressure ventilation, reducing tracheal "blow by" at peak inspiratory pressure.


**"Safe Range" window** verifies cuff pressure between 20–30cm H<sub>2</sub>O.

**Valve cap** prevents air loss during procedure.


## ORDERING INFORMATION

PressureEasy <sup>®</sup> Cuff-Pressure Controller		
Product Code	Description	Units / Case
292003	PressureEasy <sup>®</sup> Cuff-Pressure Controller with feedback line	10
292004	PressureEasy <sup>®</sup> Cuff-Pressure Controller*	10


\*Single patient use

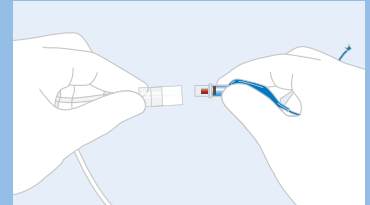


Attach PressureEasy<sup>®</sup> luer slip connector to the pilot balloon port on the tracheal tube. Ensure full insertion to prevent possible leaks.

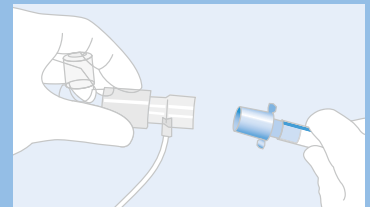

Attach high pressure adaptor

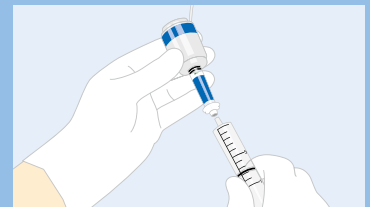

Attach syringe to stem of the PressureEasy<sup>®</sup> Cuff Pressure Controller device.

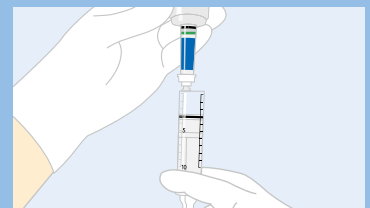

Inflate tracheal cuff until the green indicator is visible in the pressure window, indicating pressure between 20–30cm H<sub>2</sub>O.

For more information visit our website at [www.smiths-medical.com](http://www.smiths-medical.com)

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

**Smiths Medical International**  
1500 Eureka Park, Lower Pemberton  
Ashford, Kent, TN25 4BF  
Tel: +44 (0)845 850 0445

**Smiths Medical ASD, Inc.**  
6000 Nathan Lane North, Minneapolis, MN 55442, USA  
Tel: 1-614-210-7300 Toll-Free USA: 1-800-258-5361  
[www.smiths-medical.com](http://www.smiths-medical.com)

Find your local contact information at: [www.smiths-medical.com/customer-support](http://www.smiths-medical.com/customer-support)

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Portex, PressureEasy and the Smiths Medical design mark are registered trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners.

©2019 Smiths Medical. All rights reserved. RE194261GB-122019

MDBLCA1181

**Rx  
ONLY**

smiths medical