

GRIPPER PLUS®

Safety Huber Needle



Access a port

Prepare the site according to facility protocol. Access a port with the GRIPPER PLUS® safety needle at a 90° angle.



Remove clip

Remove the clip by sliding it towards the end of the needle arm and lifting.



De-access with two fingers on base

To de-access the port, approach the GRIPPER PLUS® safety needle from behind. Place fingers on the base to stabilize it.



Lift safety arm

With the other hand, place a finger on the tip of the safety arm. Lift the safety arm straight back. Notice that the needle comes out perfectly straight.



“Click” needle into locked position

Continue lifting until the needle “clicks” into the locked position. The GRIPPER PLUS® safety needle is now ready for disposal into a sharps container.

Now Available for Power Injection

GRIPPER PLUS® POWER P.A.C.

Safety Needle for Power Injection



- > **Blue tubing** provides easy identification for power
- > **Printed clamps** show power specifications



GRIPPER PLUS® POWER P.A.C. Needle Sizes

Available in non y-site and Luer-activated needless y-site (NAC)

Gauge	Length	Reorder numbers	
		Non y-site	NAC
19	3/4"	21-3368	21-3468
19	1"	21-3364	21-3469
19	1 1/4"	21-3365	21-3470
20	5/8"	21-3369	21-3464
20	3/4"	21-3367	21-3465
20	1"	21-3362	21-3466
20	1 1/4"	21-3363	21-3467

Do not power inject contrast media unless a PORT-A-CATH® POWER P.A.C. system has been verified

GRIPPER PLUS® Needle Sizes

Available in non y-site, split septum and Luer-activated needless y-site (NAC)

Gauge	Length	Reorder numbers		
		Non y-site	Y-site	NAC
19	3/4"	21-2768	21-2868	21-2968
19	1"	21-2764	21-2869	21-2969
19	1 1/4"	21-2765	21-2870	21-2970
20	5/8"	21-2769	21-2864	21-2964
20	3/4"	21-2767	21-2865	21-2965
20	1"	21-2762	21-2866	21-2966
20	1 1/4"	21-2763	21-2867	21-2967
22	5/8"	21-2770	21-2860	21-2960
22	3/4"	21-2766	21-2861	21-2961
22	1"	21-2760	21-2862	21-2962
22	1 1/4"	21-2761	21-2863	21-2963

Smiths Medical ASD, Inc.
6000 Nathan Lane North, Minneapolis, MN 55442, USA
Tel: 1-614-210-7300
Toll-Free USA: 1-800-258-5361
www.smiths-medical.com

Smiths Medical Canada Ltd.
Markham, Ontario, Canada, L3R 4Y8
Phone: 905-477-2000
Toll-Free: 1-800-387-4346

Customer and Clinical Services
U.S. 1-800-258-5361
Canada 1-800-387-4346
www.smiths-medical.com

Deltec

Product(s) described may not be licensed or available for sale in Canada or other countries outside of the United States.
GRIPPER PLUS and the Smiths Medical and Deltec design marks are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2019 Smiths Medical. All rights reserved. 9/19 VA19389-092019

CE
2797

smiths medical

GRIPPER[®] Micro

Blunt Cannula, Non-Coring Safety Needle

Insertion



Access a port

Prepare the site according to facility protocol. Holding on to the inserter as shown, insert the needle into the port at a 90° angle.



Finger on tip



Press tab in



Lift safety arm

Remove the needle

- From the back of the inserter, place fingers on each side of the inserter's base to stabilize it.
- With the other hand, place a finger on the tip of the inserter's safety arm.
- **Press the tab in** and lift the safety arm straight back until the needle **CLICKS** into the locked position.

✓ **The sharp is now removed.**



Dispose in sharps container

- Dispose of the inserter in a sharps container.
- Apply a semi-permeable dressing over the infusion site, ensuring that a minimum 4cm area surrounding the site is covered.

Removal



Remove infusion site

Place fingers on each side of the infusion site. Stabilize the port with the other hand. Lift the infusion site straight up and discard per facility protocol.

GRIPPER[®] Micro Needle Sizes

Available with a non y-site and Luer-activated needleless y-site (NAC)

Gauge	Length	Reorder numbers	
		Non y-site	NAC
19	3/4"	21-3260	21-3280
19	1"	21-3261	21-3281
19	1 1/4"	21-3262	21-3282
20	3/4"	21-3256	21-3276
20	1"	21-3257	21-3277
20	1 1/4"	21-3258	21-3278
22	3/4"	21-3251	21-3271
22	1"	21-3252	21-3272
22	1 1/4"	21-3253	21-3273

Smiths Medical ASD, Inc.
6000 Nathan Lane North, Minneapolis, MN 55442, USA
Tel: 1-614-210-7300
Toll-Free USA: 1-800-258-5361
www.smiths-medical.com

Smiths Medical is part of the global technology business Smiths Group plc.

Product(s) described may not be licensed or available for sale in Canada or other countries outside of the United States.

GRIPPER and the Smiths Medical and Deltec design marks are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2019 Smiths Medical. All rights reserved. 9/19 VA19389-092019

Smiths Medical Canada Ltd.
Markham, Ontario, Canada, L3R 4Y8
Phone: 905-477-2000
Toll-Free: 1-800-387-4346

Customer and Clinical Services
U.S. 1-800-258-5361
Canada 1-800-387-4346
www.smiths-medical.com

Deltec

CE
2797

smiths medical