

smiths medical portex[®]



EPIFUSE[®] EPIDURAL CATHETER CONNECTOR WITH NRFIT[®] CONNECTOR

Connection in a snap




SAFETY IS IN THE SNAP

The EpiFuse[®] epidural catheter connector securely locks, easily unlocks, and features a colour-coded design intended to increase safety by reducing the risk of tubing misconnections

- Secure tamper-resistant lock reduces the risk of catheter disconnection
- Accepts catheters that fit through 16G, 17G, and 18G epidural needles to allow for use with your most commonly used catheters
- MRI safe
- One-hand activation for ease-of-use and audible click when closed
- Yellow body for easy identification as an epidural connector and to help prevent misconnections
- Flat profile for increased patient comfort

EPIFUSE® EPIDURAL CATHETER CONNECTOR

EASY TO LOCK

1	Insert the exposed end of the catheter into the connector sleeve as far as possible	2	Close the connector by snapping both sides together until a clear click is heard, indicating full closure	3	The connector is now closed and the catheter secured
					

EASY TO UNLOCK

4	Firmly insert a catheter connector key into the hole on the back of the connector	5	The locking mechanism will disengage	6	Open connector and remove catheter
					

Note: The key is necessary to reopen the catheter connector. Do not dispose of the key. Keep the key with the patient until the catheter and connector are removed.

ORDERING INFO

Epifuse® Epidural Catheter Connector with Key				
Catheter Size	Connector	Connector Sleeve Colour	Product Code	Unit
19G-21G	NRFit®	Blue	24-1005-64	25/box
24G	NRFit®	White	24-1047-64	25/box

Portex® Catheter Connector Key for use with NRFit® Connectors	
Product Code	Unit
24-1055-64	50/box

For more information visit our website at www.smiths-medical.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN ALL GEOGRAPHIES

Smiths Medical ASD, Inc.
6000 Nathan Lane North
Minneapolis, MN, 55442, USA
Tel: 1-614-210-7300
Toll-Free USA 1-800-258-5361
www.smiths-medical.com

Smiths Medical Australasia Pty. Ltd.
Suite 2.03, 97 Waterloo Road
Macquarie Park NSW 2113
Tel AU: 1800 654 949

Distributed by:
USL Medical
494 Rosebank Road
Avondale, 1026 New Zealand
Tel NZ: 0800 658 814


smiths medical

Find your local contact information at: www.smiths-medical.com/customer-support/contact-us

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Epifuse, Portex and the Smiths Medical design marks are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners.
©2020 Smiths Medical. All rights reserved. IS0138.GB.AUS.Rev.A.0720