

EzPAP® Positive Airway Pressure Therapy System

Set-Up Guidelines

1 SET FLOW RATE


Begin at 5 LPM [fig 2]. Increase until desired PAP is reached. (Do not exceed 15 LPM.)

2 VERIFY PRESSURE

Should not exceed 20cm H₂O [fig 1].

3 BREATHE

Patient should be instructed to inhale and exhale slowly [fig 3].

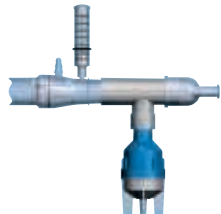

EzPAP® system
with nebulizer


NOTE: Refer to individual product Instructions For Use for warnings, cautions, contraindications, and additional information

fig 1
Gauge Guard

fig 2
Gas Supply

fig 3
Mouth Piece

7 Foot Tubing

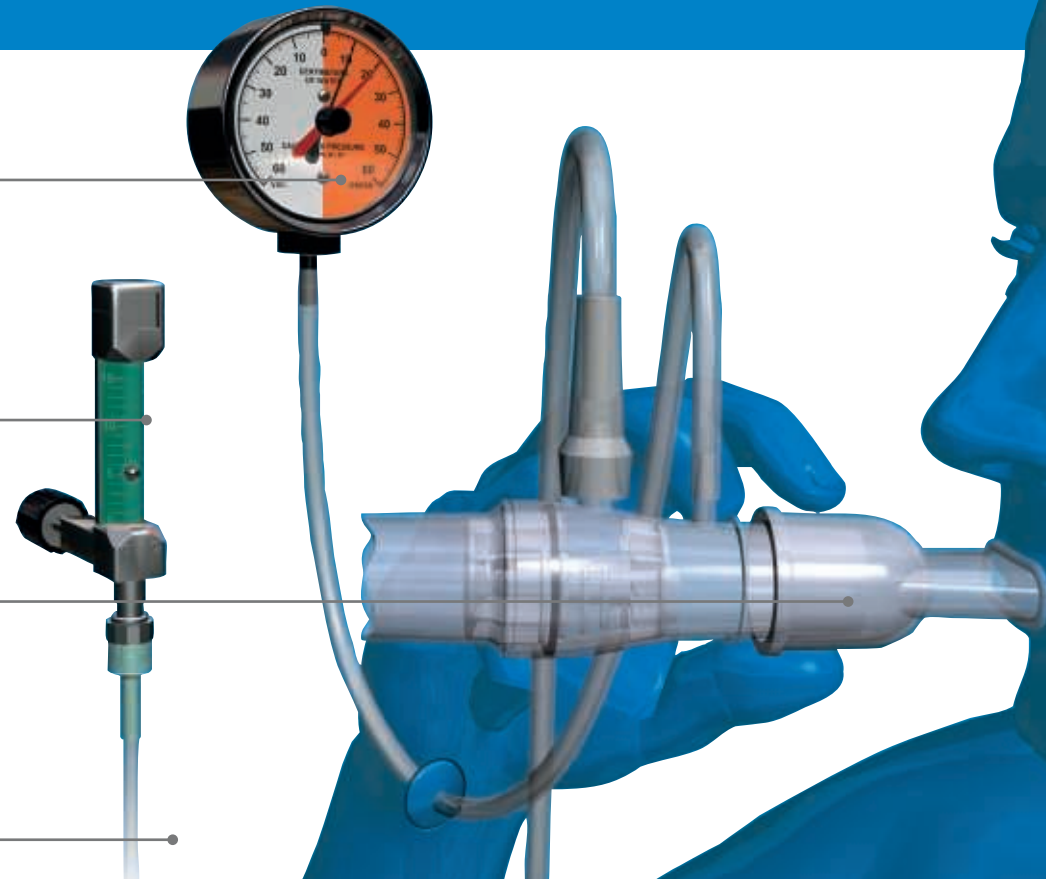

Smiths Medical International Ltd
Hythe, Kent CT21 6JL, UK
Tel: (0)1303 260551
www.smiths-medical.com

Smiths Medical ASD Inc
1500 Eureka Park, Lower Pemberton
Ashford Kent, TN25 4BF
Tel: +44 (0)845 850 0445

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

MHYTCA-1183

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician. EzPAP® and Smiths Medical design mark are trademarks of the Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. © 2016 Smiths Medical. All rights reserved. RE194235GB-102016