

EzPAP[®] Positive Airway Pressure System



The next step for atelectasis

When incentive spirometry alone won't open patients' airways, consider EzPAP[®] as your next option.

It makes providing positive airway pressure positively easy. Simply connect to a flow meter (wall air or O₂ for enhanced FiO₂), adjust to 5-15 lpm, and instruct the patient to breathe diaphragmatically through the mouthpiece or mask. No equipment to roll around. No labor-intensive CPT. No extensive training. Just a few minutes as prescribed—not for hours at a time.

EzPAP[®] features a pressure port for connection to a gauge (recommended for initial use with each patient), and a standard 22 mm OD fitting to accommodate a mouthpiece or 3 mask options.

The EASY option for atelectasis. Try it once and you'll understand.



EzPAP® Positive Airway Pressure System

Lung Expansion Therapy to Prevent or Reverse Atelectasis

EZ to meet individual patient needs.

While monitoring with a gauge, slowly adjust the flow meter until the desired airway pressures are reached. Slow inhalations and exhalations should maintain adequate airway pressure throughout the breathing cycle.

The EzPAP® Positive Airway Pressure System includes the EzPAP® lung expansion device, mouthpiece, pressure port cap, and 7 feet of tubing. Three mask options are also available.



EzPAP® may be used in conjunction with aerosol medication (e.g. nebulizer) via 22 mm connection between EzPAP® and patient.

EzPAP® Features:


-
1. **Patient end with 22 mm OD fitting**
Accommodates a mouth-piece or 3 mask sizes.
 2. **Ambient air inlet**
Has gaps to resist occlusion.
 3. **Gas inlet port**
Connects via 7 ft long tubing (included) to a wall air or O₂ flow meter.
 4. **Options**
Mask or mouthpiece available.
 5. **Pressure port with detachable cap**
Permits connection to a gauge for easy monitoring.

Product Order Chart

Each system includes EzPAP®, one 7 ft oxygen tube, one pressure port cap, and one of the following:

PRODUCT CODE	DESCRIPTION	UNITS PER CASE
23-0747	EzPAP® System with Mouthpiece	10
23-0757	EzPAP® System with Disposable Manometer and Mouthpiece	10
23-1747	EzPAP® System with Pediatric Mask	1
23-2747	EzPAP® System with Medium Mask	1
23-3747	EzPAP® System with Large Mask	1
23-6000*	EzPAP® Kit: One Pressure Gauge (With Gauge Protector), Three EzPAP® Units (with Mouthpiece), Three Gauge Guards and Ten 22 mm ID Adaptors in a Durable Plastic Box.	1

*not CE marked

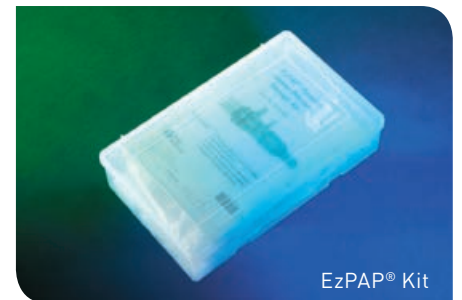

EzPAP® Kit

For more information or to order, visit www.smiths-medical.com.

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical International Ltd.
Ashford, UK TN25 4BF
Phone: +44 (0) 845 8500445
www.smiths-medical.com

Find your local contact information at: www.smiths-medical.com/customer-support

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Portex, EzPAP and the Smiths Medical and Portex design marks are registered trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. ©2013 Smiths Medical. All rights reserved. RE194239GB-082013

CE Rx
0473 ONLY

smiths medical

MROCCA-0016