


Hypodermic Needle-Pro® Fixed Needle Insulin Syringe

Step-by-Step Guidelines


IMPORTANT: For use with U-100 Insulin Only. Please refer to the Instructions for Use supplied with the product for detailed instructions, warnings, and cautions.

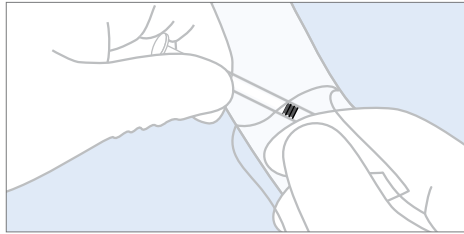

1 REMOVE DEVICE

Peel blister pouch and remove the entire device while grasping the syringe barrel. Position the Needle-Pro® device away from the needle.


Remove the cap from the needle using a “push-pull” technique.

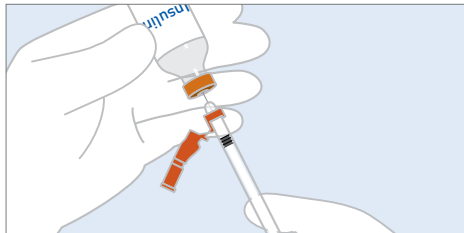

2 DRAW UP INSULIN


Draw up prescribed dose of U-100 insulin.

WARNING: For patients who mix two types of insulin, do not change the order of mixing the physician has prescribed or the model and brand of the syringe without first consulting your physician and/or pharmacist. Failure to heed this warning can result in dosage error.

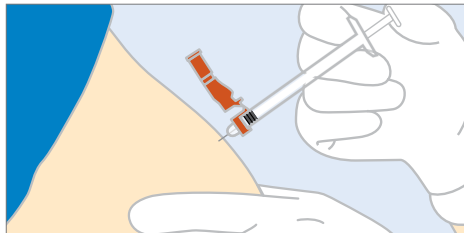

3 INJECT


Perform injection as per normal procedure.

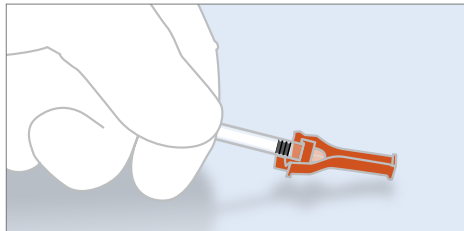

4 ENGAGE DEVICE

Upon completion of procedure, engage the Needle-Pro® device using a one-handed technique by gently pressing the sheath against a hard surface. DO NOT use free hand to press the sheath over the needle.

Visually confirm that the needle is fully engaged into the needle protection sheath.

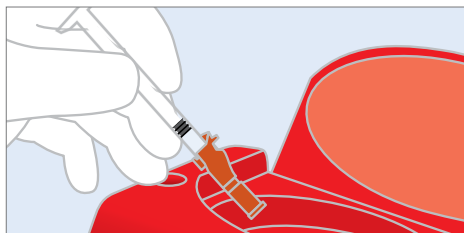

5 DISPOSE

Immediately dispose of device into an approved sharps container.

For more information visit our website at www.smiths-medical.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical ASD, Inc.
6000 Nathan Lane North
Minneapolis, MN, 55442, USA
Tel: 1-614-210-7300
Toll-Free USA 1-800-258-5361
www.smiths-medical.com

Find your local contact information at: www.smiths-medical.com/customer-support/contact-us

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Portex, Needle-Pro and the Smiths Medical design marks are trademarks of Smiths Medical. The symbol ® indicates that the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. Product referenced is CE-Marked.
© 2018 Smiths Medical. All rights reserved. SS195226EN-052018

MKEECA-0190

Rx
ONLY