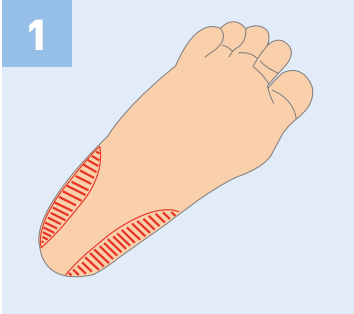
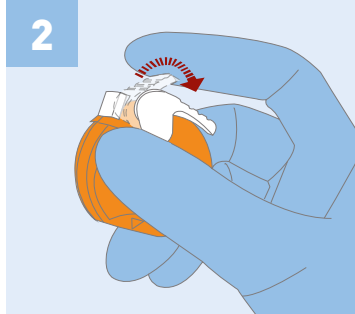
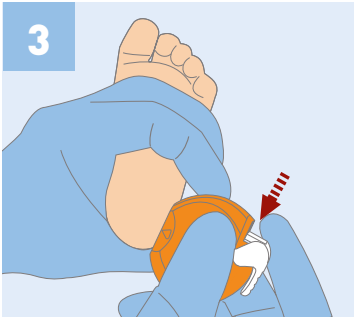
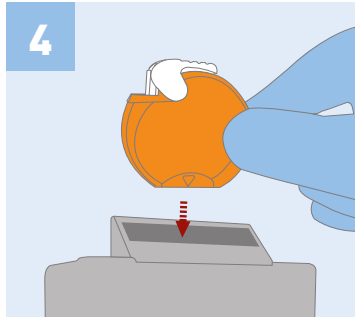
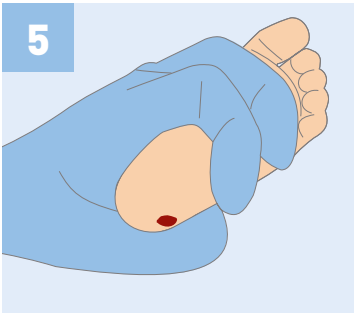



# Neoheel™ Safety Lancets

## Quick Reference Guide

 <p>1</p>	<p>Select a site identified in the shaded area. Clean the incision site of the heel with an antiseptic swab and allow skin to air dry.</p>	 <p>2</p>	<p>Release trigger lock mechanism by pushing down the safety tab.</p>
 <p>3</p>	<p>Raise the foot above the baby's heart level. Place the blade slot surface of the device in contact with the skin, and depress the trigger.</p>	 <p>4</p>	<p>After triggering, immediately remove the device from infant's heel and discard it into a biohazard sharps container.</p>
 <p>5</p>	<p>First drop of blood should be wiped away gently with a dry sterile gauze pad. Apply gentle pressure with the thumb and ease intermittently as drops of blood form.</p>	 <p>6</p>	<p>Following blood collection, gently press a dry sterile gauze pad to the incision site until bleeding has ceased.</p>

For more information visit our website at [www.smiths-medical.com](http://www.smiths-medical.com)

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

**Distributed by:**  
**Smiths Medical ASD, Inc.**  
6000 Nathan Lane North  
Minneapolis, MN 55442, USA  
Tel: +1-614-210-7300

**Smiths Medical ASD, Inc.**  
6000 Nathan Lane North  
Minneapolis, MN 55442, USA  
Tel: +1-614-210-7300

**Manufactured by:**  
**SteriLance Medical (Suzhou) Inc.**  
No. 68 Litanghe Road  
Xiangcheng, Suzhou, 215133, China

MKEECA-1618

smiths medical

Find your local contact information at: [www.smiths-medical.com/customer-support/contact-us](http://www.smiths-medical.com/customer-support/contact-us)