

# DeltaVen® Closed System Catheter


## Quick Reference Guide

### Preparation


Cleanse the insertion site per your facility's policy using the preferred skin antiseptic agent

### Insertion


**1** Check to ensure connections are secured correctly and the clamp is in the "open" position.

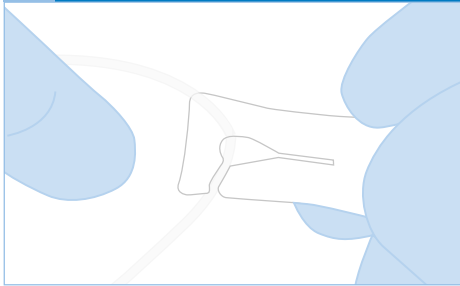


**2** Remove the sheath with a twisting motion, while holding the colored wing section. Inspect the catheter.

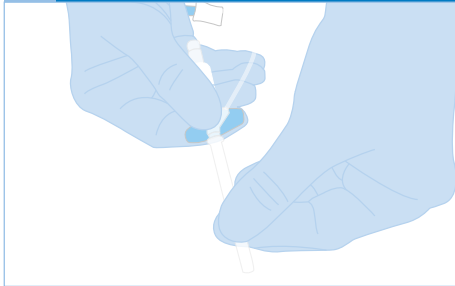


**3** Perform the venipuncture using an appropriate angle of insertion. Check for flash in the distal end of the catheter.

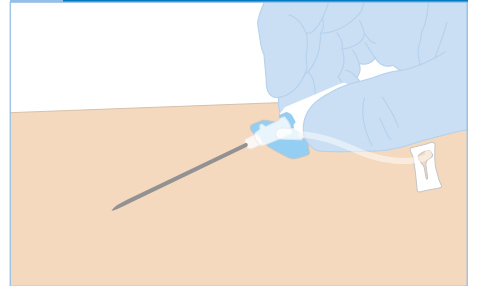


**4** **Slight push** - Lower DeltaVen® catheter near parallel to skin. Advance the whole catheter forward a few millimeters to ensure the catheter tubing is in the vein.

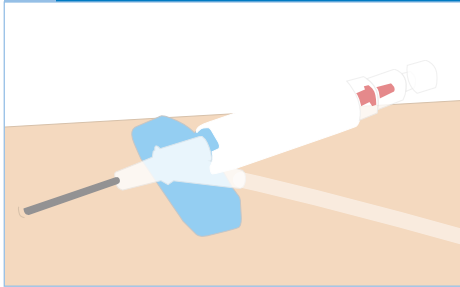


**5** **Slight pull** - Pull back on the white ribbed housing, separating the hub from the safety mechanism.

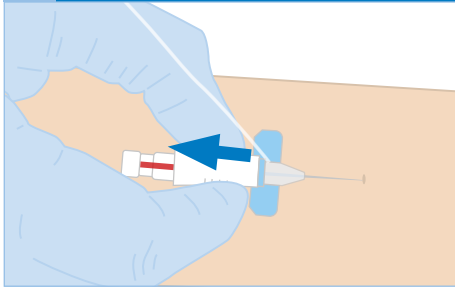


**6** **Push** - Using your finger behind the push off tab, thread the catheter into the vein holding the needle stable.

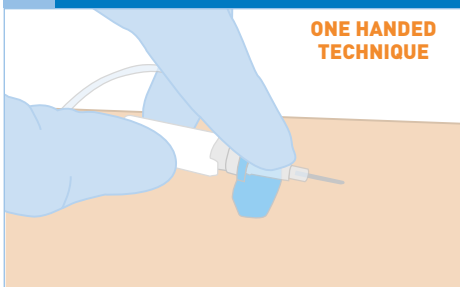

**Push** - Use your non-dominant hand to thread the catheter into the vein, holding the needle stable.

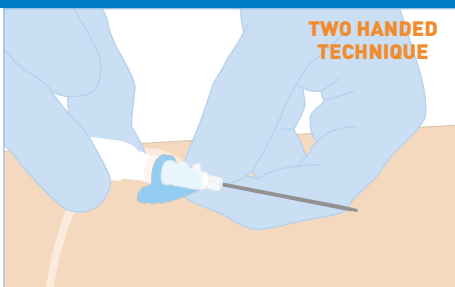

**7** **Pull** - Pull back on the white ribbed housing, near parallel to the skin, observing secondary flashback in the extension tubing. Pull until you hear an audible "click".





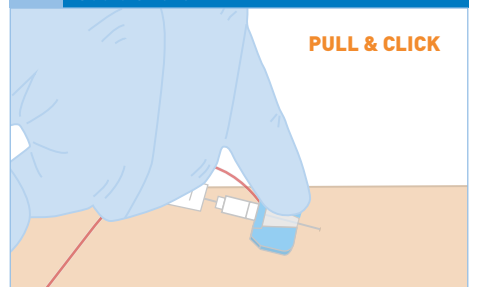


Dispose of the needle with safety mechanism in an appropriate sharps container per your facility's policies.



# DeltaVen® Closed System Catheter

## Post Insertion

### Preparation for Infusion

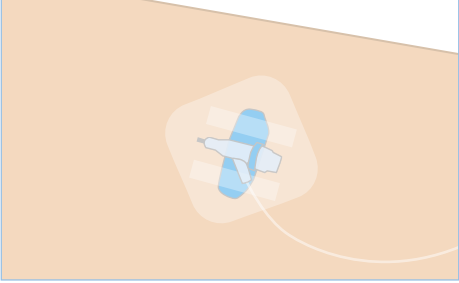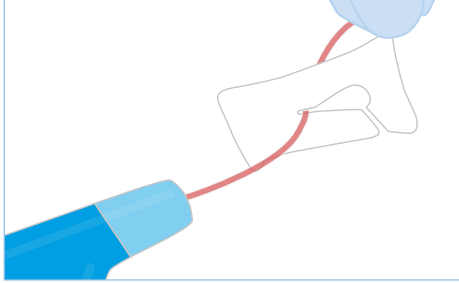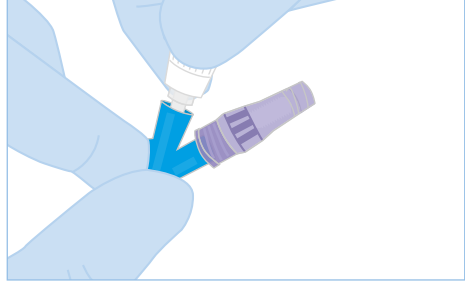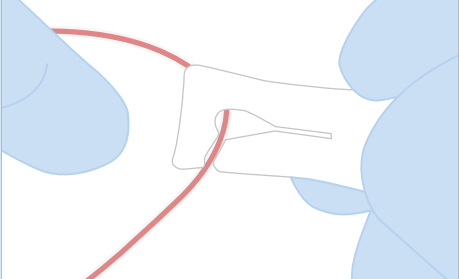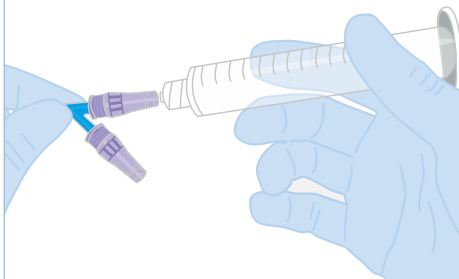- 1** Stabilize and secure the DeltaVen® catheter per your facility's protocol.  

- 2** Close the slide clamp.  

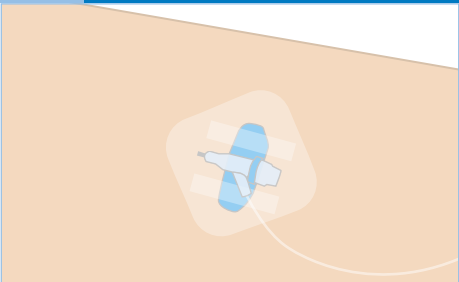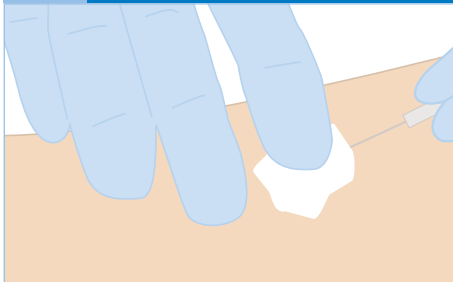- 3** Remove vent plug, replace with needleless connector or end cap.  

- 4** Open the slide clamp.  

- 5** Flush all ports, using a pulsatile flush to encourage line clearance. Close the clamp when complete.  


### Care and Maintenance

- 1** Perform a site assessment and flush the DeltaVen® catheter according to your facility's protocol.  

- 2** Remove the DeltaVen® catheter when no longer needed or clinically indicated.  


For more information visit our website at [www.smiths-medical.com](http://www.smiths-medical.com)

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

**Distributed by:**  
**Smiths Medical International**  
1500 Eureka Park, Lower Pemberton  
Ashford Kent, TN25 4BF  
Tel: +44 (0)845 850 0445  
[www.smiths-medical.com](http://www.smiths-medical.com)

**Manufactured by:**  
**Delta Med SpA**  
Via G. Rossa, 20-46019  
Viadana (Mn) - Italy  
0375 785915

MPAUC-0060

**Rx  
ONLY**

**smiths medical**

Find your local contact information at: [www.smiths-medical.com/customer-support/contact-us](http://www.smiths-medical.com/customer-support/contact-us)

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. The Smiths Medical design mark is a trademark of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. The product referenced is CE-marked. ©2017 Smiths Medical. All rights reserved. SS195486GB-122017