

DeltaVen® Tips For Success

Closed System Passive Safety IV Catheter

First Stick Success

- Seat the stabilisation platform and white ribbed housing by pressing toward each other.
- Avoid utilising only the stabilisation platform for insertion as this may cause the catheter to advance over the needle.

Easy Vein Confirmation

- Look for initial blood return in flash view chamber posterior to white ribbed needle housing.
- Advance slowly to ensure the catheter is in the vessel.
- Enhance visualisation by utilizing optional hand positions as demonstrated. (fig 1 & 2)





Threading and Disconnection

- Place fingers on ribbed needle housing, avoiding contact with the tip protector, and advance the catheter using non-dominant hand. (fig 3)
- Following completion of advancement, stabilise platform and fully retract needle maintaining the angle of insertion until audible "click." **Do not pull up during activation.** (fig 4)
- Continue to support platform and disconnect the engaged needle from the stabilisation platform by pulling with intention on the ribbed needle housing. (fig 4)



Blood Containment

- Prior to insertion, secure vent plug/end cap. (fig 5)
- Once catheter is fully threaded and retraction of needle begins, observe secondary flashback in the integrated extension tubing confirming vein placement. (fig 6)

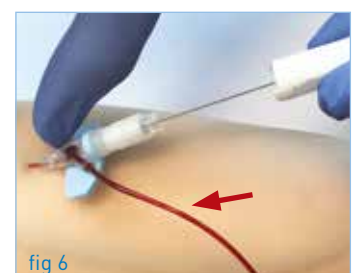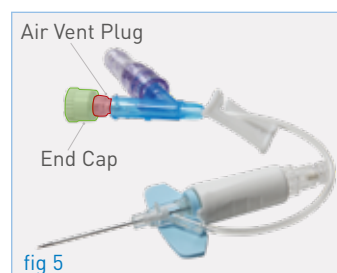

For more information visit our website at www.smiths-medical.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical Australasia Pty. Ltd.
Suite 2.03, 97 Waterloo Road
Macquarie Park NSW 2113
Ph AU: 1800 654 949
www.smiths-medical.com

Distributed By:
USL Medical
494 Rosebank Road
Avondale, 1026 New Zealand
Ph NZ: 0800 658 814

Manufactured by:
Delta Med SpA
Via G. Rossa, 20-46019
Viadana (Mn) - Italy
0375 785915

MMSPCA-0431

**Rx
ONLY**
smiths medical

Find your local contact information at: www.smiths-medical.com/customer-support/contact-us

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. The Smiths Medical design mark are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. ©2019 Smiths Medical. All rights reserved. SS195514GBAUS-012019