

SERIVA™ IV catheter


Guide for Successful Insertion




1 Hold the Seriva IV catheter as shown and apply pressure on the edge of the protective cap. The cap will detach with this single-handed technique.




2 With the index finger on the push off tab, penetrate the vein while holding the catheter as shown in the picture.

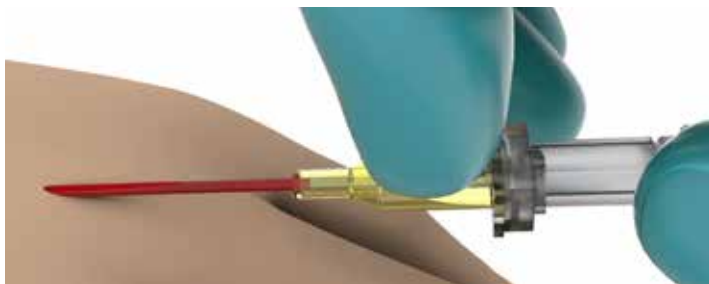

3 Once the needle enters the vein, flashback will be seen along the catheter, through the FLASH-VUE® catheter window that allows early flashback visualization.

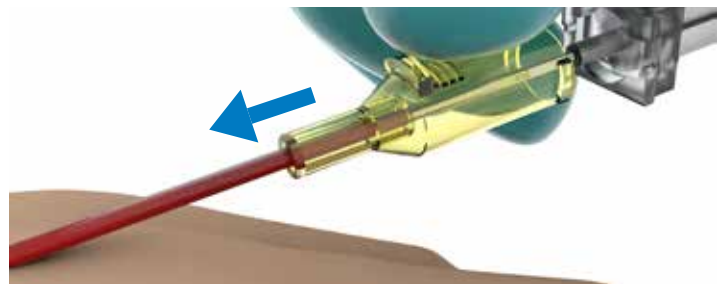

4 With index finger on the push off tab, advance the catheter forward into the vein.

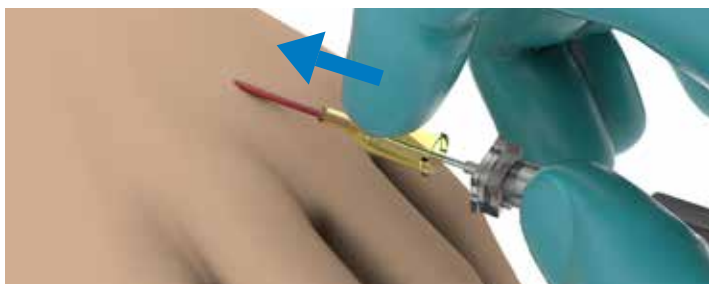

5 Continue to advance catheter using the push off tab until the whole catheter is in the vein.



Please scan the code above to access the full video.

For more information visit our website at www.smiths-medical.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical ASD, Inc.
6000 Nathan Lane North
Minneapolis, MN 55442, USA
Phone: 1-219-989-3297
Toll-Free USA 1-866-216-8808
www.smiths-medical.com

Find your local contact information at: www.smiths-medical.com/customer-support/contact-us

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Jelco and the Smiths Medical design mark are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. There is no connection between Smiths Medical and Intuitive Surgical. The product referenced is CE-marked. ©2018 Smiths Medical. All rights reserved. SS195522GBSEA-082018

MKEECA-0182