

Quick Reference Guide

V1030

SurgiVetTM

Rx ONLY






For use by or on the order of a licensed veterinarian.




Attention, see operation manual for additional instructions for use.


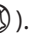
SurgiVet® V1030 Hand Held Pulse Oximeter Simplified Operation Guide

Monitor Operating Keys

-  **Alarm Silence**
Use this key to silence the alarms and to cancel the alarm silence.
-  **Menu / Enter Key**
Press this key to enter the menu system, and to advance to the next menu selection.
-  **Cancel / Exit Key**
Press this key to exit the menu system.
-  **Up and Down Arrows**
Use the up and down arrow keys to adjust pulse beep volume during normal operation. In a menu, use these keys to adjust the selection.
-  **ON/OFF Key**
Momentarily press this key when the device is OFF to turn the monitor ON.
Press and hold this key when the device is OFF to clear the patient trend data.

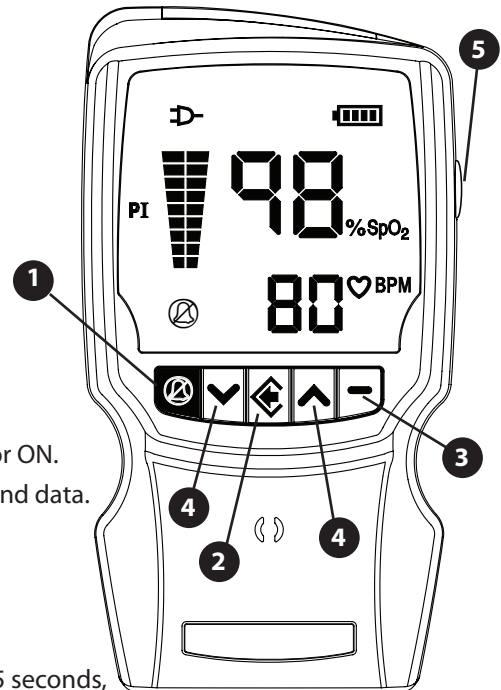







Silencing Alarms

- Audible alarms can be silenced for two (2) minutes.
- To silence the alarm tone, press the **Alarm Silence** key () .
- The Alarm Silence indicator will light solid for the first minute and 45 seconds, then flash the last 15 seconds.
- To cancel the 2-minute silence, press the **Alarm Silence** key () .

NOTE: The Alarm Silence key will not affect visual alarm indicators or pulse tone.

NOTE: Permanent Alarm Silence is not available.

Changing the Alarm Limits

- To select alarm limit defaults by patient type, press the **Menu** key () once to display the current patient type setting. *PI* will be displayed as well as the current setting: Horse (*Hr S*), Dog (*dOG*), Cat (*CAt*), Factory (*FAL*), or Custom (*CU5*).
- Use the **Up** and **Down** keys () to select the desired patient type.
- Press the **Menu** key () again to display the desired alarm limit values for that patient type.
- Use the **Up** and **Down** keys () to increase or decrease the setting.
- Press the **Menu** key () to advance to the next alarm limit.
- To exit, press the **Exit** key () or wait 20 seconds until the patient reading is displayed.

smiths medical

Pulse Volume

1. Use **Up** and **Down** keys (^ ▼) to raise or lower pulse beep volume.
2. Continue to press the **Down** key to silence the pulse tone completely.

Entering Setup Menu

1. Power oximeter on by pressing the **On/Off** key (%) located on the device's upper right side.
2. When the patient number (Example: P 1) is displayed, press and hold the **Exit** key (■) and then press the **Menu** key (◀).
3. The Averaging Menu will be the first option. $\overline{SpO_2}$ will be displayed. Release the **Exit** key (■).

Setting Averaging

1. While $\overline{SpO_2}$ is being displayed, use the **Up** and **Down** keys (^ ▼) to select the desired SpO₂ averaging (1-4).
2. Press the **Menu** key (◀) to advance to the next menu.
3. To exit menus, press the **Exit** key (■) or wait 20 seconds and menu will be exited automatically.

Setting Sensitivity

1. Advance from previous menu or see *Entering Setup Menu*. HS (for High Sensitivity) will be displayed.
2. Use the **Up** and **Down** keys (^ ▼) to turn the High Sensitivity setting **On** or **OFF**.

Display Brightness

1. Advance from previous menu or see *Entering Setup Menu*. The display will show db.
2. Use the **Up** and **Down** keys (^ ▼) to select the desired brightness, ranging from 1-10. The default setting is 10 (brightest).


Setting the Date

1. Advance from previous menu or see *Entering Setup Menu*. To skip initial menus, if setting only date, press **Menu** key (◀) three (3) times until dt 1 (Date Setting 1) is displayed.
2. Use the **Up** and **Down** keys (^ ▼) to select the year. Range is 00 (2000) to 99 (2099).
3. Press the **Menu** key (◀) to advance to the Month Menu. The display will show dt 2 (Date Setting 2).
4. Use the **Up** and **Down** keys (^ ▼) to select the month. Range is 01 (January) to 12 (December).
5. Press the **Menu** key (◀) to advance to the Day Menu. The display will show dt 3 (Date Setting 3).
6. Use the **Up** and **Down** keys (^ ▼) to select the day. Range is from 01 to 31.

Setting the Time

1. Advance from previous menu or see *Entering Setup Menu*. To skip initial menus, if setting only time, press **Menu** key (◀) six (6) times until t 1 (Time Setting 1) is displayed.
2. The display will show t 1. Use the **Up** and **Down** keys (^ ▼) to select the hour. Range is 00 (Midnight) to 23 (11PM).
3. Press the **Menu** key (◀) to advance to Minute Menu. The display will show t 2. Range is 0-59.
4. Press **Exit** key (■) to exit Setup Menus.

 **Attention, see operation manual for Warnings, Cautions, Notes and additional instructions for use.**

Manufactured By: Smiths Medical PM, Inc.
 N7W22025 Johnson Drive
 Waukesha, Wisconsin 53186-1856 USA
 Phone: 1-262-513-8500
 Toll-Free: (U.S.A. only) 1-888-745-6562
 Fax: 1-262-542-0718
 www.surgivet.com

SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.

©2010 Smiths Medical Family of companies. All rights reserved.