

The Effect of Short-Term Periods of Pre-operative Warming in the Prevention of Peri-operative Hypothermia

Clinical study

AUTHORS

E.-P. Horn, B. Bein, R. Böhm, M. Steinfath, N. Sahili and J. Höcker

CENTER AND COUNTRY

Kiel University, Germany

TYPE OF STUDY

Proposed, Randomized

STUDY OBJECTIVE

The aim of the study was to evaluate the performance of different durations of active pre-operative skin-surface warming (pre-warming) to prevent peri-operative hypothermia and postoperative shivering. 200 patients were randomly assigned, scheduled for surgery of 30-90 minutes under general anesthesia, to receive passive insulation or forced-air skin surface warming for 10, 20 or 30 minutes.

METHODOLOGY

200 healthy adult patients undergoing elective surgery under general anesthesia: laparoscopic cholecystectomy; inguinal hernia repair; breast surgery; minor orthopaedic surgery; and ENT surgery with expected duration > 30 minutes, but < 90 minutes. Patients were not studied if they were < 18 years old, classified as ASA physical status 3 or higher or planned for combined general/regional anaesthesia. Body temperature was measured at the

tympanic membrane. In accordance with the current guidelines, hypothermia was defined as a core temperature 36°C/96.8°F.

Shivering was graded by visual inspection. Postoperatively, shivering was graded by an investigator blinded to patients' core temperatures and group assignments using a four-point scale (0 = no shivering; 1 = intermittent, low-intensity shivering; 2 = moderate shivering; 3 = continuous, intense shivering).

STUDY RESULTS

There were significant difference in changes of core temperature between the non-pre-warmed group and all the pre-warmed groups ($p < 0.00001$), but none between the three pre-warmed groups ($p = 0.54$).

Without pre-warming, 38/55 (69%) of patients became hypothermic (<36°C/96.8°F) at the end of anaesthesia, whereas only 7/52 (13%), 3/43 (7%) and 3/50 (6%) of patients following 10, 20 or 30 minutes pre-warming, respectively, became hypothermic ($p < 0.001$ vs no pre-warming).

Shivering was observed in 10/55 (18%) of patients without pre-warming, and in 3/52 (6%), 3/43 (7%) and 1/50 (2%) of patients with following 10, 20 or 30 minute pre-warming, respectively ($p = 0.02$ vs no pre-warming).

	No pre-warming (n = 55)	10 minutes of pre-warming (n = 52)	20 minutes of pre-warming (n = 43)	30 minutes of pre-warming (n = 50)
Active warming required during surgery	37 (67%)	16 (31%) ¹	1 (2%) ^{1,2}	3 (6%) ^{1,2}
Hypothermic at start of PACU	38 (69%)	7 (13%)	3 (7%)	3 (6%)
Shivering in PACU	10 (18%)	3 (6%) ¹	3 (7%) ¹	1 (2%) ¹

PACU, postoperative care unit; ¹p < 0.05 vs 0 minutes warming; ²p < 0.05 vs 10 minute warming.

DISCUSSION

Core temperature of patients who were not pre-warmed declined more than with pre-warming, despite active warming during the procedure. In the pre-warmed groups, the need for intra- and post-operative warming was lower, and the duration was shorter than in the patients who were not pre-warmed. Starting active warming intra-operatively for the first time after the core temperature has already decreased below <36°C/96.8°F does not reduce or prevent further hypothermia.

FULL ARTICLE AVAILABLE AT

<http://onlinelibrary.wiley.com/doi/10.1111/i.1365-2044.2012.07073.x/full>

CONCLUSION

Pre-warming of patients before general anesthesia prevents hypothermia and reduces shivering. In this study, pre-warming time did not influence changes of core temperature.

PUBLICATION OF PRESENTATION

Presented in part at the American Society of Anaesthesiologists Annual Meeting, Chicago, USA; October 2011. Published in the Journal of the Association of Anaesthesiologists of Great Britain and Ireland, Volume 67, Issue 6; June 2012.

SMITHS MEDICAL PRODUCTS USED

Level 1® Equator® convective warmer
YSI 400
Snuggle Warm® upper body blankets

YEAR OF PUBLICATION

2012

Smiths Medical has no financial or personal relationship with the author(s) that might bias or be seen to bias their opinions. Smiths Medical provides this summary solely as a service to medical professionals. Smiths Medical did not participate in this study and has not had full access to all of the data in this study and takes no responsibility for the integrity of the data and the accuracy of the data analysis.

For more information visit our website at www.smiths-medical.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

MPAUC-1794

Smiths Medical ASD, Inc.
6000 Nathan Lane North
Minneapolis, MN 55442, USA
Tel: 1-614-210-7300
Toll-Free USA: 1-800-258-5361

smiths medical

Find your local contact information at: www.smiths-medical.com/customer-support

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. The Smiths Medical design mark is trademark of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. Product referenced is CE marked. ©2017 Smiths Medical. All rights reserved. TM194908EN-112017