

Effectiveness of Strategies for the Management and/or Prevention of Hypothermia within the Adult Peri-operative Environment

Clinical study

AUTHORS

Sandeep Moola BDS MHSM, Craig Lockwood RN BN
GradDipClinNurs MNSc

CENTER AND COUNTRY

University of Adelaide, Australia

TYPE OF STUDY

Retrospective, studies review

STUDY OBJECTIVE

The objective of this review was to identify the most effective methods for the treatment and/or preventions of hypothermia in intraoperative or postoperative patients.

METHODS

Nineteen studies with a combined 1,451 patients who underwent different surgical procedures were included in this review. Meta-analysis was not possible. Two independent reviewers assessed methodological validity of papers selected for retrieval and any disagreements were resolved through discussion.

STUDY RESULTS

Forced-air warming in pregnant women scheduled for caesarean delivery under regional anesthesia prevented maternal and fetal hypothermia. Extra warming with forced air compared to routine thermal care was effective in reducing the incidence of surgical wound infections and postoperative cardiac complications. Passive warming with reflective heating blankets or elastic bandages wrapped around the legs tightly were found to be ineffective in reducing the incidence or magnitude of hypothermia.

CONCLUSION

There are significant benefits associated with forced-air warming. Evidence supports commencement of active warming preoperatively and monitoring it throughout the intraoperative period. Single strategies such as forced-air warming were more effective than passive warming; however, combined strategies, including preoperative commencement, use of warmed fluids plus forced-air warming as other active strategies were more effective in vulnerable groups (age or durations of surgeries).

ABSTRACT AVAILABLE AT

<https://www.ncbi.nlm.nih.gov/pubmed/22093385>

YEAR OF PUBLICATION

2011

For more information visit our website at www.smiths-medical.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical International
1500 Eureka Park, Lower Pemberton
Ashford Kent, TN25 4BF
Tel: +44 (0)845 850 0445

MPAUC-1796

smiths medical

Find your local contact information at: www.smiths-medical.com/customer-support

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. The Smiths Medical design mark is trademark of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. Product referenced is CE marked. ©2017 Smiths Medical. All rights reserved. TM194910GB-112017