

smiths medical
portex®

Portex® Blue Line Ultra® Suctionaid®

Airway Management: For Easier Tracheostomy Maintenance



BLUE LINE ULTRA[®] SUCTIONAID[®]

A new addition to the Blue Line Ultra[®] range, the Blue Line Ultra[®] Suctionaid[®] is ideal for improving patient well-being by maintaining a clean, hygienic and unobstructed airway. It also has the ability to remove secretions from above the cuff.

- Reduces the risk of infection
- Reduces the risk of aspiration
- Maintains patient comfort
- Has all the benefits of Blue Line Ultra[®] including Soft Seal cuff - low pressure, high volume to minimise risk of tracheal trauma while maintaining an effective seal

Reducing the potential risk of infection

Accumulation and stasis of contaminated mucus and subglottic secretions above the cuff of tracheostomy tubes can be uncomfortable for patients and provides an ideal growth medium for pathogens. The integrated suction lumen of Blue Line Ultra[®] Suctionaid[®] allows removal of these pooled secretions.

Reducing the risk of aspiration

However good the cuff, microaspiration of contaminated material can potentially lead to pulmonary infection. Maintaining tracheostomy hygiene and regular removal of secretions with Blue Line Ultra[®] Suctionaid[®] can help reduce this aspiration.


Maintaining patient comfort

Blue Line Ultra[®] Suctionaid[®] makes it easier to maintain patient comfort.


- Inner Cannula – can be removed, cleaned and re-inserted without disturbing the tracheostomy tube itself
- Tracheostomy tube may be left in situ for up to 30 days reducing the frequency of tube changes
- Suction lumen integrated into tube wall for smooth, easy insertion to minimise trauma
- Tube material softens at body temperature to conform to patient anatomy
- Clear, flexible flange for comfort and aesthetic acceptability




Suction Above the Cuff - Suctionaid®

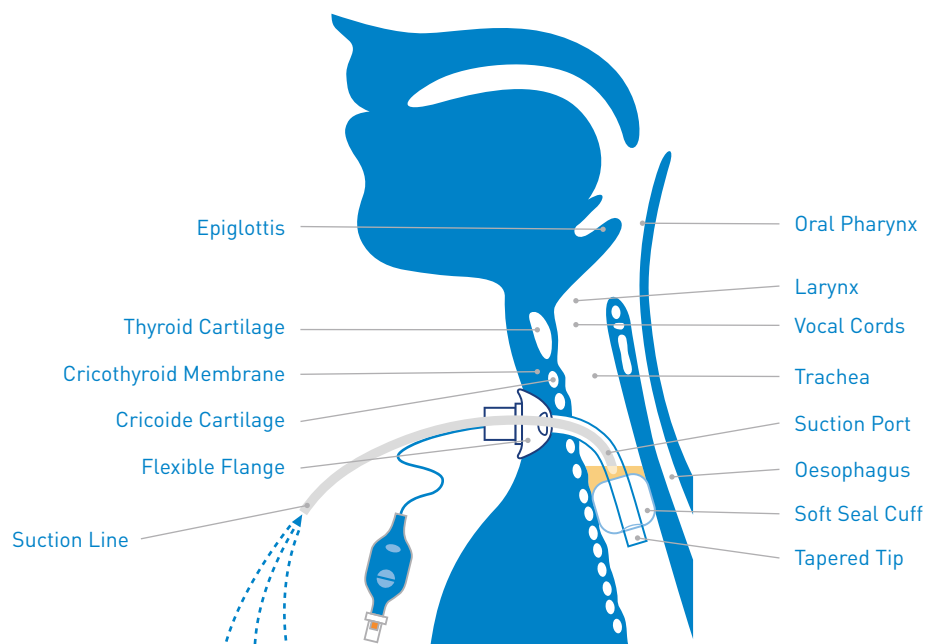


Syringe Aspiration

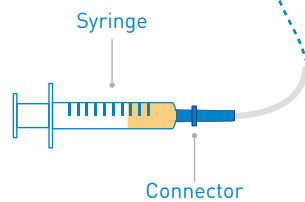

Intermittent Suction

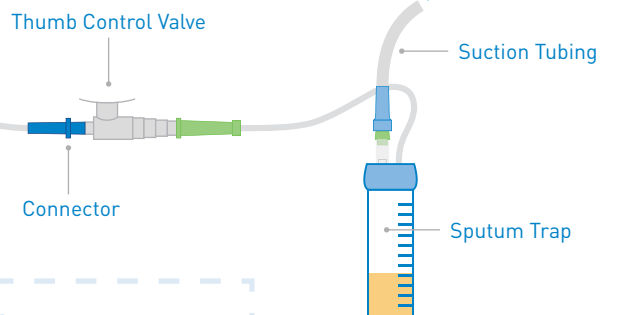

Continuous Suction

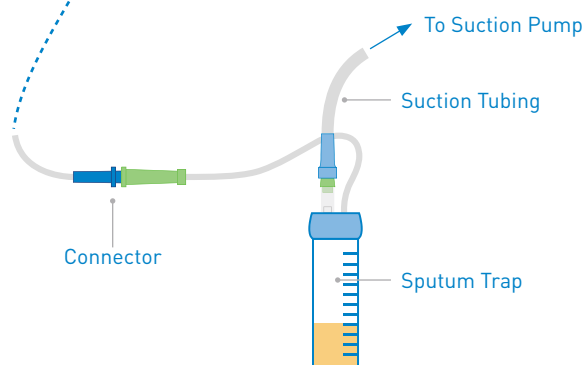

When using continuous or intermittent suctioning, use low level suction up to a maximum of 20 mmHg



100/860

Blue Line Ultra® Suctionaid® Tracheostomy Tube

Product Code	Tube Size	Tube I.D.	Tube O.D.	Tube Length	Units Per Case
100/860/060	6.0 mm	6.0 mm	9.2 mm	64.5 mm	10
100/860/070	7.0 mm	7.0 mm	10.5 mm	70.0 mm	10
100/860/075	7.5 mm	7.5 mm	11.3 mm	73.0 mm	10
100/860/080	8.0 mm	8.0 mm	11.9 mm	75.5 mm	10
100/860/085	8.5 mm	8.5 mm	12.6 mm	78.0 mm	10
100/860/090	9.0 mm	9.0 mm	13.3 mm	81.0 mm	10
100/860/100	10.0 mm	10.0 mm	14.0 mm	87.5 mm	10



100/870

Blue Line Ultra® Suctionaid® Tracheostomy Tube Kit

Product Code	Tube Size	Tube I.D.	Tube O.D.	Tube Length	Units Per Case
100/870/060	6.0 mm	6.0 mm	9.2 mm	64.5 mm	1
100/870/070	7.0 mm	7.0 mm	10.5 mm	70.0 mm	1
100/870/075	7.5 mm	7.5 mm	11.3 mm	73.0 mm	1
100/870/080	8.0 mm	8.0 mm	11.9 mm	75.5 mm	1
100/870/085	8.5 mm	8.5 mm	12.6 mm	78.0 mm	1
100/870/090	9.0 mm	9.0 mm	13.3 mm	81.0 mm	1
100/870/100	10.0 mm	10.0 mm	14.0 mm	87.5 mm	1

Includes inner cannula with disconnection wedge, tracheostomy tube holder, cleaning brush, and patient label.



100/563/070

Blue Line Ultra® Suctionaid® Percutaneous Dilator Tracheostomy Kit with Single Stage Dilator

Product Code	Tube Size	Tube I.D.	Tube O.D.	Tube Length	Units Per Case
100/563/070	7.0 mm	7.0 mm	10.5 mm	70.0 mm	1
100/563/080	8.0 mm	8.0 mm	11.9 mm	75.5 mm	1
100/563/090	9.0 mm	9.0 mm	13.3 mm	81.0 mm	1

Includes single stage dilator, introducer with handle, and procedural components.



100/891

Blue Line Ultra® Suctionaid® Percutaneous Tracheostomy Kit with Forceps

Product Code	Tube Size	Tube I.D.	Tube O.D.	Tube Length	Units Per Case
100/891/070	7.0 mm	7.0 mm	10.5 mm	70.0 mm	1
100/891/080	8.0 mm	8.0 mm	11.9 mm	75.5 mm	1
100/891/090	9.0 mm	9.0 mm	13.3 mm	81.0 mm	1



100/893

Blue Line Ultra® Suctionaid® Percutaneous Tracheostomy Kit without Forceps

Product Code	Tube Size	Tube I.D.	Tube O.D.	Tube Length	Units Per Case
100/893/070	7.0 mm	7.0 mm	10.5 mm	70.0 mm	1
100/893/080	8.0 mm	8.0 mm	11.9 mm	75.5 mm	1
100/893/090	9.0 mm	9.0 mm	13.3 mm	81.0 mm	1

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical India Private Limited

508-509, Western Edge 2,
W.E. Highway, Borivali (E),
Mumbai- 400066, India
T: +91 22 28705210 F: +91 22 28705211
E: csindia@smiths-medical.com
www.smiths-medical.com

MMSPCA-0565

Find your local contact information at: www.smiths-medical.com/customer-support/contact-us

smiths medical

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Portex, Blue Line Ultra, Suctionaid and the Smiths Medical design mark are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trade names, trademarks or service marks of their respective owners.
©2018 Smiths Medical. All rights reserved. TR194408GBIN-122018.