



# Portex® Blue Line Ultra® Tracheostomy Tube

## Inner Cannula Replacement

### Removal

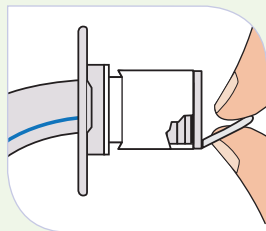


Disconnect equipment from 15 mm connector if present. Grasp the "ring pull."

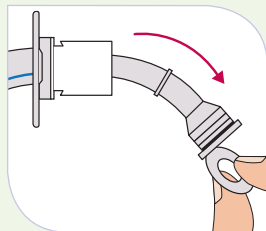


Remove the inner cannula, pulling in a down and outward motion, as indicated.

### Insertion

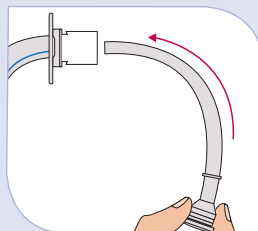

Remove the correct size inner cannula from the sterile package. Grasp as shown and gently insert into the tracheostomy tube.

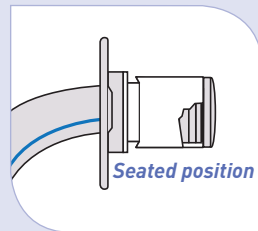

Gently seat the inner cannula into the 15 mm connector. A light "snap" will indicate proper position. Reconnect equipment and check for security if necessary.

## Blue Line Ultra® Tracheostomy Tube Ordering Chart

| Style                                | Cuffed                                    | Uncuffed                   | Tube Inner Diameter (mm) | Inner Cannula Diameter (mm) | Tube Outer Diameter (mm) | Length (mm) | Cuff Resting Diameter (mm) | Inner Cannula              | Jackson/Shiley Equivalent |
|--------------------------------------|---|----------------------------|--------------------------|-----------------------------|--------------------------|-------------|----------------------------|----------------------------|---------------------------|
| Regular Fenestrated Suctionaid® Tube | 100/815/060<br>100/817/060<br>100/875/060 | 100/816/060<br>100/818/060 | 6.0                      | 5.0                         | 9.2                      | 64.5        | 20.0                       | 100/856/060<br>100/858/060 | 4                         |
| Regular Fenestrated Suctionaid® Tube | 100/815/070<br>100/817/070<br>100/875/070 | 100/816/070<br>100/818/070 | 7.0                      | 6.0                         | 10.5                     | 70.0        | 24.0                       | 100/856/070<br>100/858/070 | None                      |
| Regular Suctionaid® Tube             | 100/815/075<br>100/875/075                | 100/816/075                | 7.5                      | 6.5                         | 11.3                     | 73.0        | 30.0                       | 100/856/075<br>100/858/075 | None                      |
| Regular Fenestrated Suctionaid® Tube | 100/815/080<br>100/817/080<br>100/875/080 | 100/816/080<br>100/818/080 | 8.0                      | 7.0                         | 11.9                     | 75.5        | 30.0                       | 100/856/080<br>100/858/080 | 6                         |
| Regular Suctionaid® Tube             | 100/815/085<br>100/875/085                | 100/816/085                | 8.5                      | 7.5                         | 12.6                     | 78.0        | 30.0                       | 100/856/085<br>100/858/085 | None                      |
| Regular Fenestrated Suctionaid® Tube | 100/815/090<br>100/817/090<br>100/875/090 | 100/816/090<br>100/818/090 | 9.0                      | 8.0                         | 13.3                     | 81.0        | 30.0                       | 100/856/090<br>100/858/090 | 8                         |
| Regular Fenestrated Suctionaid® Tube | 100/815/100<br>100/817/100<br>100/875/100 | 100/816/100<br>100/818/100 | 10.0                     | 9.0                         | 14.0                     | 87.5        | 30.0                       | 100/856/100<br>100/858/100 | 10                        |

### Smiths Medical ASD, Inc.

Norwell, MA 02061, USA  
Phone: 1-214-618-0218  
Toll-Free USA: 1-800-258-5361  
[www.smiths-medical.com](http://www.smiths-medical.com)

### Smiths Medical Canada Ltd.

Markham, Ontario, Canada, L3R 4Y8  
Phone: 905-477-2000  
Toll-Free: 1-800-387-4346

### Customer and Clinical Services

U.S. 1-800-258-5361  
Canada 1-800-387-4346  
[www.smiths-medical.com](http://www.smiths-medical.com)

Smiths Medical is part of the global technology business Smiths Group plc.

Product[s] described may not be licensed or available for sale in Canada or other countries outside of the United States.

Blue Line Ultra and the Portex and Smiths Medical design marks are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2011 Smiths Medical. All rights reserved. 4/11 TR194433EN

**smiths medical**