

smiths medical
deltec®



GRIPPER MICRO®
Blunt Cannula,
Non-Coring
Safety Needle

A revolutionary design that
fits your patient's lifestyle



GRIPPER MICRO® Safety Needle

The unique safety and comfort features of the popular GRIPPER PLUS® needle have now been incorporated into the new GRIPPER MICRO® blunt cannula, non-coring safety needle, along with several new additions that make it a good choice for small patients or for those requiring long-term or repeated infusions.

The new GRIPPER MICRO® safety needle is a two-piece system that includes an inserter and an infusion site. After the needle is inserted into an implanted port, the inserter (sharp) is removed, leaving behind a small, low profile infusion site with a blunt cannula. The needle's unique safety arm provides an audible CLICK when it is lifted for extra assurance that the needle is locked into a safe position. And the infusion site's small size and foam pad help provide increased patient comfort for long-term therapy.

GRIPPER MICRO® SAFETY NEEDLE OFFERS:

- Small, low profile infusion site for ease in dressing
- Nonabsorbent foam pad for extra patient comfort
- Non-sharp blunt cannula for safe removal and disposal following infusion
- Trocar, non-coring needle that locks into place for sharps disposal

ORDERING INFORMATION

GRIPPER MICRO® Safety Needle			
Gauge	Length	Reorder Numbers Non Y-site	Reorder Numbers NAC
19	3/4 inch	21-3260-24	21-3280-24
19	1 inch	21-3261-24	21-3281-24
19	1 ¼ inch	21-3262-24	21-3282-24
20	3/4 inch	21-3256-24	21-3276-24
20	1 inch	21-3257-24	21-3277-24
20	1 ¼ inch	21-3258-24	21-3278-24
22	3/4 inch	21-3251-24	21-3271-24
22	1 inch	21-3252-24	21-3272-24
22	1 ¼ inch	21-3253-24	21-3273-24

ACTUAL SIZE
SMALLER THAN A DIME



EASY AND SAFE INSERTION

1 Insert needle into port.



2 Press tab in and lift safety arm until it clicks, then dispose of inserter and safe needle into a sharps container.



3 All that's left is a small, low profile infusion site.



For more information visit our website at www.smiths-medical.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical ASD, Inc.
6000 Nathan Lane North
Minneapolis, MN, 55442, USA
Tel: 1-614-210-7300
Toll-Free USA 1-800-258-5361
www.smiths-medical.com

Find your local contact information at: www.smiths-medical.com/customer-support/contact-us

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. GRIPPER, PORT-A-CATH, and Deltec design marks are trademarks of Smiths Medical. The symbol © indicates it is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are trademarks or service marks of their respective owner. ©2018 Smiths Medical. All rights reserved. VA192501EN-062018

MMSPCA-0293

Rx
ONLY

smiths medical