

GRIPPER Plus®

Safety Huber Needles

1 Access a port



Prepare the site according to facility protocol. Access a port with the GRIPPER PLUS® safety needle at a 90° angle.

2 Remove clip



Remove the clip by sliding it towards the end of the needle arm and lifting.

3 De-access with two fingers on base



To de-access the port, approach the GRIPPER PLUS® safety needle from behind. Place fingers on the base to stabilize it.

4 Lift safety arm



With the other hand, place a finger on the tip of the safety arm. Lift the safety arm straight back. Notice that the needle comes out perfectly straight.

5 "Click" needle into locked position



Continue lifting until the needle "clicks" into the locked position. The GRIPPER PLUS® safety needle is now ready for disposal into a sharps container.

GRIPPER PLUS® Power P.A.C.

Safety Needle for Power Injection

- Blue tubing provides easy identification for power
- Printed clamps show power specifications



GRIPPER PLUS® POWER P.A.C.

Gauge	Length	Reorder Numbers	
		Non y-site	NAC
19	3/4"	21-3368-24	21-3468-24
19	1"	21-3364-24	21-3469-24
19	1 1/4"	21-3365-24	21-3470-24
20	5/8"	21-3369-24	21-3464-24
20	3/4"	21-3367-24	21-3465-24
20	1"	21-3362-24	21-3466-24
20	1 1/4"	21-3363-24	21-3467-24

Needle sizes available in non y-site and Luer-activated needleless y-site (NAC)

Do not power inject contrast media unless a PORT-A-CATH® POWER P.A.C. system has been verified

GRIPPER PLUS® Needle Sizes

Gauge	Length	Reorder Numbers		
		Non y-site	Y-site	NAC
19	3/4"	21-2768-24	21-2868-24	21-2968-24
19	1"	21-2764-24	21-2869-24	21-2969-24
19	1 1/4"	21-2765-24	21-2870-24	21-2970-24
20	5/8"	21-2769-24	21-2864-24	21-2964-24
20	3/4"	21-2767-24	21-2865-24	21-2965-24
20	1"	21-2762-24	21-2866-24	21-2966-24
20	1 1/4"	21-2763-24	21-2867-24	21-2967-24
22	5/8"	21-2770-24	21-2860-24	21-2960-24
22	3/4"	21-2766-24	21-2861-24	21-2961-24
22	1"	21-2760-24	21-2862-24	21-2962-24
22	1 1/4"	21-2761-24	21-2863-24	21-2963-24

Available in non y-site, split septum and Luer-activated needleless y-site (NAC)

Gripper Micro®

Blunt Cannula, Non-Coring Safety Needles

Insertion



Access a port

Prepare the site according to facility protocol. Holding on to the inserter as shown, insert the needle into the port at a 90° angle.


GRIPPER MICRO® non coring safety needle

- Small, low profile infusion site for ease in dressing
- Non absorbent foam pad for extra patient comfort
- Blunt end for safer disposal on removal



Remove the needle

- From the back of the inserter, place fingers on each side of the inserter's base to stabilize it.
- With the other hand, place a finger on the tip of the inserter's safety arm.
- Press the tab in and lift the safety arm straight back until the needle CLICKS into the locked position.
- The sharp is now removed.

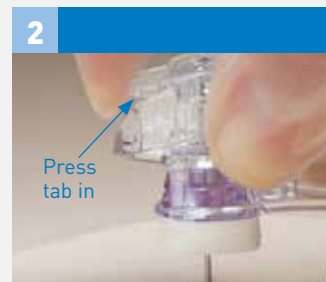

Dispose



Dispose in sharps container

- Dispose of the inserter in a sharps container.
- Apply a semi-permeable dressing over the infusion site, ensuring that a minimum 4cm area surrounding the site is covered.

Removal



Remove infusion site

Place fingers on each side of the infusion site. Stabilize the port with the other hand. Lift the infusion site straight up and discard per facility protocol.

Gripper Micro® Needle Sizes

Gauge	Length	Reorder Numbers	
		Non y-site	NAC
19	¾"	21-3260-24	21-3280-24
19	1"	21-3261-24	21-3281-24
19	1¼"	21-3262-24	21-3282-24
20	¾"	21-3256-24	21-3276-24
20	1"	21-3257-24	21-3277-24
20	1¼"	21-3258-24	21-3278-24
22	¾"	21-3251-24	21-3271-24
22	1"	21-3252-24	21-3272-24
22	1¼"	21-3253-24	21-3273-24

Available with a non y-site and Luer-activated needleless y-site (NAC)

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical Australasia Pty Ltd
Suite 201, Level 2
10 Norbrik Drive
Bella Vista NSW 2153, Australia
Tollfree - Australia 1800 654 949
www.smiths-medical.com

Distributed by
USL Medical
494 Rodebank Road
Avondale, 1062 New Zealand
Freecall: 0800 658 814

MMSPCA-0129

**Rx
ONLY**

smiths medical

Find your local contact information at: www.smiths-medical.com/customer-support/contact-us

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. Gripper Plus, Gripper Micro, Deltec and the Smiths Medical design marks are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. 2018 Smiths Medical. All rights reserved. VA193389GBAUS-022018