


Portex[®] Filter-Pro[®] Air-Bubble-Removal Device


Step-by-Step Guidelines

The Filter-Pro[®] device enables you to expel air bubbles from an arterial blood sample safety without risk of exposure to blood or blood-borne pathogens.

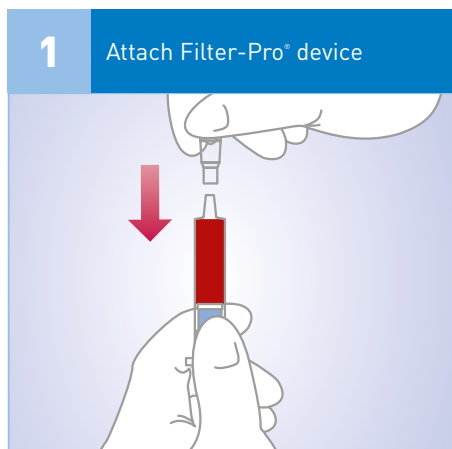


- Push the Filter-Pro[®] device onto the syringe Luer, ensuring it is secure.

Warning: Maximum blood volume in 1ml syringe should not exceed 0.5ml to prevent plunger dislodgement.

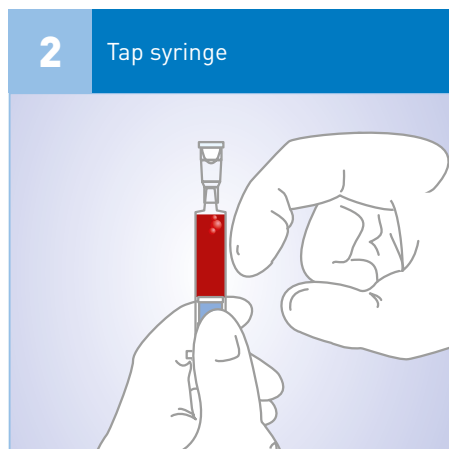

- Hold the syringe Luer-end upwards and tap to move the air bubbles to the top.

Warning: Keep syringe upright before air bubbles are removed. The filter seals upon contact with blood and may prevent air bubble removal in step 3.

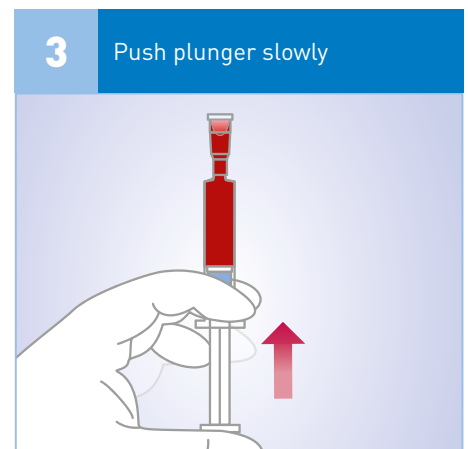

- Push the plunger slowly to expel air from the sample. The filter seals on contact with blood allowing the Filter-Pro[®] device to act as a transport cap.
- Mix the sample by holding the syringe and rotating your wrist back and forth for 20 to 30 seconds.
- Label and transport. Testing should be performed within 30 minutes of collecting.



Scan to watch the Filter-Pro[®] video

Minimum Blood Volume	
1ml syringe	3ml syringe
0.25ml	0.8ml

For more information visit our website at www.smiths-medical.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical India Private Limited
 508-509, Western Edge 2,
 W.E. Highway, Borivali (E),
 Mumbai- 400066, India
 T: +91 22 28705210 F: +91 22 28705211
 E: csindia@smiths-medical.com
www.smiths-medical.com

Find your local contact information at: www.smiths-medical.com/customer-support/contact-us

Smiths Medical is part of the global technology business Smiths Group plc. Portex, Filter-Pro and the Portex and Smiths Medical design marks are trademarks of Smiths Medical. The symbol © indicates that the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. Product referenced is CE-Marked. © 2019 Smiths Medical. All rights reserved. SS195221GBSEA-072019

MKEECA-0172

**Rx
ONLY**

smiths medical