

GRIPPER PLUS®

Safety Huber Needle



Access a port

Prepare the site according to facility protocol. Access a port with the GRIPPER PLUS® safety needle at a 90° angle.



Remove clip

Remove the clip by sliding it towards the end of the needle arm and lifting.



De-access with two fingers on base

To de-access the port, approach the GRIPPER PLUS® safety needle from behind. Place fingers on the base to stabilize it.



Lift safety arm

With the other hand, place a finger on the tip of the safety arm. Lift the safety arm straight back. Notice that the needle comes out perfectly straight.



“Click” needle into locked position

Continue lifting until the needle “clicks” into the locked position. The GRIPPER PLUS® safety needle is now ready for disposal into a sharps container.

Now Available for Power Injection

GRIPPER PLUS® POWER P.A.C.

Safety Needle for Power Injection



- > **Blue tubing** provides easy identification for power
- > **Printed clamps** show power specifications



GRIPPER PLUS® POWER P.A.C. Needle Sizes

Available in non y-site and Luer-activated needless y-site (NAC)

| Gauge | Length | Reorder numbers | |
|-------|--------|-----------------|---------|
| | | Non y-site | NAC |
| 19 | 3/4" | 21-3368 | 21-3468 |
| 19 | 1" | 21-3364 | 21-3469 |
| 19 | 1 1/4" | 21-3365 | 21-3470 |
| 20 | 5/8" | 21-3369 | 21-3464 |
| 20 | 3/4" | 21-3367 | 21-3465 |
| 20 | 1" | 21-3362 | 21-3466 |
| 20 | 1 1/4" | 21-3363 | 21-3467 |

Do not power inject contrast media unless a PORT-A-CATH® POWER P.A.C. system has been verified

GRIPPER PLUS® Needle Sizes

Available in non y-site, split septum and Luer-activated needless y-site (NAC)

| Gauge | Length | Reorder numbers | | |
|-------|--------|-----------------|---------|---------|
| | | Non y-site | Y-site | NAC |
| 19 | 3/4" | 21-2768 | 21-2868 | 21-2968 |
| 19 | 1" | 21-2764 | 21-2869 | 21-2969 |
| 19 | 1 1/4" | 21-2765 | 21-2870 | 21-2970 |
| 20 | 5/8" | 21-2769 | 21-2864 | 21-2964 |
| 20 | 3/4" | 21-2767 | 21-2865 | 21-2965 |
| 20 | 1" | 21-2762 | 21-2866 | 21-2966 |
| 20 | 1 1/4" | 21-2763 | 21-2867 | 21-2967 |
| 22 | 5/8" | 21-2770 | 21-2860 | 21-2960 |
| 22 | 3/4" | 21-2766 | 21-2861 | 21-2961 |
| 22 | 1" | 21-2760 | 21-2862 | 21-2962 |
| 22 | 1 1/4" | 21-2761 | 21-2863 | 21-2963 |

Smiths Medical ASD, Inc.
St. Paul, MN 55112, USA
Phone: 1-214-618-0218
Toll-Free USA: 1-800-258-5361
www.smiths-medical.com

Smiths Medical Canada Ltd.
Markham, Ontario, Canada, L3R 4Y8
Phone: 905-477-2000
Toll-Free: 1-800-387-4346



EC Authorized Representative
Smiths Medical International Ltd.
WD24 4LG, UK
Phone: +44 (0) 1923 246434

Customer and Clinical Services
U.S. 1-800-258-5361
Canada 1-800-387-4346
www.smiths-medical.com

Deltec

smiths medical