

**smiths medical**

bringing technology to life



## Safe-T-Lance® Plus Safety Lancet – Push Button Activated

Safe. Comfortable. Effective.



## 3 EASY STEPS



**1.** Twist off the protective cap, and then pull it straight out.



**2.** Press the Safe-T-Lance® Plus Safety Lancet body firmly against the puncture site and press the button to activate the device.



**3.** Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

*WARNING: Do not use if protective cap has been previously removed from lancet. Clean the skin according to your safety procedures. Discard used lancet into a sharps container in accordance with facility guidelines, national and state regulations.*

## FEATURES AND BENEFITS

### SAFETY

- Single use only
- Automatic needle retraction and lock out
- Needlestick injury and cross-infection prevention

### EFFICACY







- Wide product range – available in 6 different sizes
- Derives proper amount of blood for different capillary tests

### COMFORT

- Easy to use in just three steps
- Designed to control needle depth
- Double spring action
- Silicone coated needles
- Ergonomic grip for precise handling



### Safe-T-Lance® Plus Lancet

	Paediatric 1028-1	1.5mm blade	1.2mm depth		Normal Flow 1014-1	21G needle	1.8mm depth
	Max Flow 1005-1	1.5mm blade	1.6mm depth		Low Flow 1020-1	25G needle	1.4mm depth
	High Flow 1008-1	18G needle	1.8mm depth		Micro Flow 1025-1	28G needle	1.6mm depth

Ordering Quantity: 200 Units/Box, 2000 Units/Case

For more information visit our website at [www.smiths-medical.com](http://www.smiths-medical.com)

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

**Distributed by:**  
Smiths Medical ASD, Inc.  
Keene, NH 03431, USA  
Tel: +1-603-352-3812

**Smiths Medical International**  
1500 Eureka Park, Lower Pemberton  
Ashford Kent, TN25 4BF  
Tel: +44 (0)845 850 0445

**Manufactured by:**  
HTL-STREFA S.A.  
ul. Adamówek 7  
95-035 Ozorków POLAND

**EC Authorized Representative**  
Smiths Medical International Ltd.  
TN25 4BF, UK  
Tel: +44 (0) 1233 722100

MKEECA-0140

Find your local contact information at: [www.smiths-medical.com/customer-support/contact-us](http://www.smiths-medical.com/customer-support/contact-us)

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use for a complete listing of warnings and precautions. The Smiths Medical design mark is a trademark of Smiths Medical. Safe-T-Lance is a registered trademark of HTL-STREFA S.A. Product referenced is CE-Marked. ©2017 Smiths Medical. All rights reserved. SS195429GB-042017

smiths medical